



Mast Group Ltd.
Mast House, Derby Road,
Bootle, Merseyside, L20 1EA

United Kingdom
Tel: + 44 (0) 151 472 1444
Fax: + 44 (0) 151 944 1332
email: sales@mast-group.com
Web: www.mast-group.com



Mast Diagnostica GmbH Feldstrasse 20 DE-23858 Reinfeld

Tel: + 49 (0) 4533 2007 0 Fax: + 49 (0) 4533 2007 68 email: mast@mast-diagnostica.de Web: www.mast-group.com

Germany

Mast Diagnostic

12 rue Jean-Jacques Mention CS91106, 80011 Amiens, CEDEX 1 France

Tél: + 33 (0) 3 22 80 80 67 Fax: + 33 (0) 3 22 80 99 22 email: info@mast-diagnostic.fr Web: www.mast-group.com



Actinomycete MAST® SELECTATAB

MS25 Series

Intended Use

For the selective isolation of actinomycetes.

FOR IN VITRO DIAGNOSTIC USE ONLY

Contents

25 (small) or 10 (large) MAST® SELECTATAB. See pack label.

Formulation

Material:	Concentration in medium:
Metronidazole	2.5 mg/L
Nalidixic Acid	25 mg/L

Storage and shelf life

Store unopened at 2 to 8°C until the expiry date shown on the pack label. Once opened, store MAST® SELECTATAB in capped, original packaging at 2 to 8°C until the expiry date shown on the pack label.

Precautions

For *in vitro* diagnostic use only. Observe approved biohazard precautions and aseptic techniques. To be used only by adequately trained and qualified laboratory personnel. Sterilise all biohazard waste before disposal. Refer to Product Safety Data sheet.

Materials required but not provided

Standard microbiological supplies and equipment such as loops, MAST® culture media, swabs, applicator sticks, incinerators and incubators, etc., as well as serological and biochemical reagents and additives such as blood.

Procedure

- 1. Label Petri dishes using the self-adhesive labels provided.
- Sterilise the appropriate volume of MAST® Columbia Agar (DM115D), cool to 50 to 55°C and stand in a water bath at this temperature.
- 3. Using sterile forceps, add one MAST® SELECTATAB to the volume of medium as stated on the pack label and label the bottle. Allow to stand for several minutes at 50 to 55°C in the water bath until the MAST® SELECTATAB has broken up.
- 4. After the MAST® SELECTATAB has broken up, swirl 3 to 4 times and invert to complete dispersal. An alternative method is to first dissolve the MAST® SELECTATAB in 3 to 5 mL of sterile water and add this to the appropriate volume of medium.

- 5. Supplement the medium with 10% sterile defibrinated horse blood, mix thoroughly and pour culture plates (15 to 20ml per plate) and allow to set.
- 6. Prepared culture plates may be used immediately or stored in plastic bags at 2 to 8°C for up to one week before use.
- Spread inoculate over the surface of separate dried plates 0.1mL of each sample dilution as prepared below
- 8. Prepare samples for inoculation in the following manner. Place swab, coil (Intrauterine Contraceptive Device) or any pus from the coil into 5 mL MAST® Thioglycollate Broth (DM220D) and from this primary dilution make tenfold dilutions (10⁻¹ to 10⁻⁴) in Thioglycollate Broth. Prior dilution of the sample is important making it easier to identify small actinomycete colonies.
- 9. Plates should be incubated anaerobically in 10% CO₂ and examined after 4, 10 and 14 days.

Interpretation of results

Actinomyces spp. appear as irregular shaped white colonies approximately 2 to 4 mm in diameter, which are very adherent and difficult to emulsify.

Quality control

Check for signs of deterioration. Quality control must be performed with at least one organism to demonstrate a positive reaction and at least one organism to demonstrate a negative reaction. Do not use the product if the reactions with the control organisms are incorrect. The list below illustrates a range of performance control strains which the end user can easily obtain.

Test Organisms	Result
Staphylococcus aureus ATCC® 25923	No growth
Proteus mirabilis ATCC® 29906	No growth
Candida albicans ATCC® 90028	Growth
Bacteroides fragilis ATCC® 25285	No growth
Clostridium perfringens ATCC® 13124	No growth
Actinomyces israelii ATCC® 10049	Growth

References

Bibliography available on request.