

MAST® ID ODC Agar

IDM30

Intended use

An agar medium for demonstration of amino acid decarboxylase.

Contents

See pack label.

Formulation*

Material:	Concentration in medium:
Dextrose	4.0 g/litre
Yeast extract	3.0 g/litre
Bacteriological peptone	1.0 g/litre
L-ornithine hydrochloride	7.0 g/litre
Bromo-cresol purple	0.02 g/litre
Agar	24.0 g/litre
Final pH: 6.3 ± 0.2	

Storage and shelf life

All dehydrated culture media containers should be kept tightly closed and stored in a dry place at 10 to 25°C until the expiry date shown on the pack label.

Precautions

For *in vitro* diagnostic use only. Observe approved hazard precautions and aseptic techniques. To be used only by adequately trained and qualified laboratory personnel. Sterilise all biohazard waste before disposal. Refer to Product Safety Data sheet (available on request or via MAST® website).

Materials required but not provided

Standard microbiological supplies and equipment such as loops, MAST® selective supplements, swabs, applicator sticks, incinerators and incubators, etc., as well as serological and biochemical reagents and additives such as blood.

Procedure

1. Refer to pack label for quantities and volumes required. Prepare MAST® ID/ODC Agar (IDM30/A) by suspending the powder in distilled or deionised water. For sachet packs, dissolve the entire contents of the sachet in the volume shown on the label.
2. Sterilise by autoclaving at 121°C (15 p.s.i.) for 15 minutes. Do not overheat any carbohydrate-containing medium.
3. Mix well and pour culture plates (15 to 20ml per plate) into Petri dishes which have been labelled using the self-adhesive labels provided. Self adhesive labels are provided in each box of preweighed sachets.
4. Prepared culture plates may be used immediately or stored in plastic bags at 2 to 8°C for up to one week.

5. Prepare a suspension of each organism equivalent in density to a 0.5 McFarland standard. Inoculate the surface of a well dried plate using a replicating device, e.g. the SCANURIDOT Multipoint Inoculator, to deliver each inoculum onto the agar surface.
6. Allow the inoculum drops to dry before disturbing and incubate plates aerobically for 18 to 24 hours at 35 to 37°C.

Interpretation of results

After incubation record growth and colour development in the medium. A positive result indicating decarboxylation of ornithine is shown by a purple colour in the growth spot and surrounding medium i.e. no change of colour is positive. Negative results show yellow growth and yellow in surrounding around the colony.

Quality control

Check for signs of deterioration. Quality control must be performed with at least one organism to demonstrate expected performance. Do not use the product if the result with the control organism is incorrect. The list below illustrates a range of performance control strains which the end user can easily obtain.

Test Organisms	
<i>Citrobacter freundii</i> ATCC® 8090	Negative
<i>Salmonella typhimurium</i> ATCC® 14028	Positive

References

Bibliography available on request.