



Mast Group Ltd.
Mast House, Derby Road, Bootle
Liverpool, Merseyside, L20 1EA
United Kingdom
Tel: + 44 (0) 151 472 1444
Fax: + 44 (0) 151 944 1332
email: sales@mast-group.com
Web: www.mast-group.com

Mast Diagnostica GmbH
Feldstrasse 20
DE-23858 Reinfeld
Germany
Tel: + 49 (0) 4533 2007 0
Fax: + 49 (0) 4533 2007 68
email: mast@mast-diagnostica.de
Web: www.mast-group.com

Mast Diagnostic
12 rue Jean-Jacques Mention
CS91106, 80011 Amiens, CEDEX 1
France
Tél: + 33 (0) 3 22 80 80 67
Fax: + 33 (0) 3 22 80 99 22
email: info@mast-diagnostic.fr
Web: www.mast-group.com



Sodium Desoxycholate

RM60

Intended use

A standardised selective inhibitor for enteric pathogens.

Contents

See pack label.

Storage and shelf life

All dehydrated culture media containers should be kept tightly closed and stored in a dry place at 10 to 25°C until the expiry date shown on the pack label.

Precautions

For *in vitro* diagnostic use only. Observe approved hazard precautions and aseptic techniques. To be used only by adequately trained and qualified laboratory personnel. Sterilise all biohazard waste before disposal. Refer to Product Safety Data sheet (available on request or via MAST® website).

Since the late 1980s MAST® has sourced all its animal-derived culture media ingredients from non-bovine animals wherever possible. All animal-derived materials, including bovine materials, are from BSE-free regions of the world, are from animals certified as disease-free by qualified veterinarians and have been heat-treated in accordance with European regulations. Despite these precautions, MAST® Media Raw Materials must not be used in the manufacture of vaccines or food ingredients, or in the manufacture of any other high-risk products involving culture processes such as those destined for *in-vivo* or agricultural use.

Materials required but not provided

Standard microbiological supplies and equipment such as loops, MAST® selective supplements, swabs, applicator sticks, incinerators and incubators, etc., as well as serological and biochemical reagents and additives such as blood.

Description

MAST® Sodium Desoxycholate (RM60A) is soluble in water giving a clear, colourless solution. A specific bile acid, derived from deconjugated bile salts, it is an extremely pure form, which has been accurately standardised to give good inhibition of non-enteric species.

Procedure

MAST® Sodium Desoxycholate should be added at the appropriate concentration to culture media with additional ingredients e.g. agar, meat extracts and peptones as specified in the formulation being prepared.

Quality control

Check for signs of deterioration. Quality control must be performed with at least one organism to demonstrate expected performance. Do not use the product if the result with the control organism is incorrect. The list below illustrates a range of performance control strains which the end user can easily obtain.

Test Organisms	Result
<i>Escherichia coli</i> ATCC® 25922	Growth*
<i>Staphylococcus aureus</i> ATCC® 25923	Inhibition*

*Nutrient preparation with the addition of 2% w/v MAST® Sodium Desoxycholate.

References

Bibliography available on request.