

Mast Uri[®] Locate Reagent

Statement of purpose

Orientation marker as an accessory to Mast Uri[®] Plates and Mast Uri[®] Plus.

Technical Data

Description

Mast **Uri**[®] *Locate* is a solution used to indicate that Mast **Uri**[®] *Plates* have been inoculated and will act as an orientation marker for the Mast **Uri**[®] *Plus* to ensure that the plate has been placed in the correct orientation.

Product Formulation

A liquid reagent containing a non-hazardous commercially formulated red ink with fluorescein dye in a white plastic dropper bottle.

Interpretation

Mast **Uri**[®] *Plates* can be read on the Mast **Uri**[®] *Plus*. The system will indicate if the plate has been orientated incorrectly by alerting the user to re-insert the plate in the correct orientation.

Storage and Shelf Life

All containers should be kept tightly closed and stored in a dark, dry place at 25°C or below until the expiry date shown on the pack label.

Quality Control

MPM-CON can be used to validate that the reagent is performing as expected. If the reagent is not recognized when the plate is correctly orientated, then the plate set can be read with the Mast **Uri**[®] *Locate* function disabled and the manufacturer must be notified of the fault.

Limitations of Use

The reagent is light sensitive and can degrade if exposed to white light for an extended period of time.

United Kingdom

Mast Group Ltd. Mast House Derby Road, Bootle Merseyside L20 1EA

Tel. +44 (0) 151 933 7277 Fax +44 (0) 151 944 1332 Email: sales@mastgrp.com

Germany

Mast Diagnostica GmbH Feldstrasse 20 DE-23858 Reinfeld

Tel. +49 (0)4533 2007 0 Fax +49 (0)4533 2007 68 Email: mast@mast-diagnostica.de

115

France

Mast Diagnostic 115 Rue Jules Barni 80011 Amiens CEDEX 1

Tel. +33 (0) 322 80 80 67 Fax +33 (0) 322 80 99 22 Email: info@mast-diagnostic.fr

www.mastgrp.com