

Mast House, Derby Road, Bootle, Merseyside L20 1EA, United Kingdom. Tel: +44 (0)151 933 7277 Fax: +44 (0)151 944 1332 www.mastgrp.com

Syphilis Screen Recombinant

PRINCIPLE OF THE TEST

Syphilis Screen Recombinant is an ELISA kit based on the direct sandwich method for the detection of total antibodies against *Treponema pallidum* in serum or plasma. During incubation, the antibodies present in the sample are immunologically coupled to the solid phase antigen as well as to the antigen labelled with horseradish peroxidase (HRP), creating an antigen-antibody-antigen-HRP "sandwich". The antigen is constituted of immunodominant recombinant proteins of *T. pallidum*. Washing eliminates the unbound conjugate and the bound enzymatic activity is determined by adding a chromogenic substrate. The blue reaction which develops turns yellow on addition of the stop solution (see figure over the page). The intensity of the colour is proportional to the antibody concentration in the sample.

BENEFITS OF THE KIT

Syphilis Screen Recombinant has been designed to be particularly "user-friendly" and easy to handle.

• Sensitivity: 100%, Specificity: 99.7%. Reference: H. Young et al. (2000) *Int J STD & AIDS* 11: 288-291.

• Ready to use stable, liquid reagents:

The conjugate (recombinant antigen labelled with HRP), substrate (TMB) and STOP solution are liquid and ready to use. The reagents are stable up to expiry date, once opened, if stored at 2 - 8 °C after use.

- No sample pre-dilution
- Results in 40 minutes 30 minutes sample incubation, 10 minutes enzymatic reaction.
- One-step procedure

Ordering Information:

PRODUCT CODE: 91100 PACK SIZE: 192 Tests



Mast House, Derby Road, Bootle, Merseyside L20 1EA, United Kingdom. Tel: +44 (0)151 933 7277 Fax: +44 (0)151 944 1332 www.mastgrp.com



Schematic Diagram of Recombinan Syphilis

