ENZYWELL

RUBELLA IgG AVIDITY

REF 91095 (24 tests)

Manufactured by: DIESSE Diagnostica Senese
Via delle Rose 10
53035 Monteriggioni (Siena) - Italy

CE
0123

INDICE / INDEX

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17. BIBLIOGRAFIA

1. S. Rousseau, K. Hedman. Rubella infection and reinfection distinguished by avidity of IgG. Lancet 1: 1108 (1988).
2. H. Thomas, C. Morgan. Rubella-specific IgG subclass avidity ELISA and its role in the differentiation between primary Rubella and rubella infection. Epidemiol. Infect. 101: 591 (1988).
3. L. Matter, K. Kogelschats, D. Germann. Serum levels of rubella virus antibodies indicating immunity: response to vaccination of subjects with low or undetectable antibody concentrations. J. Inf. Dis. (1997).

DIESSE Diagnostica Senese
Via delle Rose 10
53035 Monteriggioni (Siena) Italy.
Tel. 0577-587111

INSTRUCTIONS FOR USE

**ENZYWELL
RUBELLA IgG AVIDITY KIT**

REF 91095

(English)

1. INTENDED USE:

IMMUNOENZYMATIC METHOD FOR THE DETERMINATION OF THE AVIDITY OF IgG-CLASS ANTIBODIES TO RUBELLA IN HUMAN SERUM

2. SUMMARY AND EXPLANATION OF THE TEST:

Rubella virus causes a very common infectious disease in humans, accompanied by a typical diffused rash. The course of the illness is generally short and without complications. However, it is harmful for the fetus if contracted during pregnancy, because the virus is teratogenic and may cause very serious malformations especially during the first month of pregnancy. It is therefore essential to determine the patient's state of immunity before the start of pregnancy, if possible, in order to perform vaccination or, if this is impossible, to monitor the course of the pregnancy to observe whether seroconversion occurs.

Sometimes it is not sufficient to test for IgM antibodies, as the presence of this class may be due to the persistence of IgM or asymptomatic reinfection by Rubella, without risk for the fetus.

For this reason it is useful to assay the avidity of the IgG antibodies, in as far as it has been demonstrated that avidity is low in the acute phase of infection and increases over a period of time. The presence of low avidity antibodies is therefore an indication of a recent or current infection. The avidity of IgG antibodies in the serum can be assayed with this ELISA kit.

3. PRINCIPLE OF THE TEST:

The test is based on the ELISA technique (Enzyme linked Immunosorbent Assay).

The purified and inactivated Rubella virus antigen is bound to the solid phase. The specific immunoglobulins are bound to the antigen through incubation with dilute human serum. The sample is placed in two wells (or in four wells if the test is performed in duplicate) and left to incubate.

After washing of the solid phase, the Avidity Buffer is placed in one well and the rinsing buffer in the other. The avidity buffer will cause dissociation of the antigen/antibody bond, proportional to the avidity of the antibodies. After washings to eliminate the proteins that have not reacted, incubation is performed with the conjugate composed of anti-human IgG monoclonal antibodies labelled with peroxidase HRP.

The unbound conjugate is eliminated and the peroxidase substrate specific for the enzyme is added.

The colour which develops is proportional to the concentration of specific antibodies present in the serum sample still bound to the antigen. The ratio between the OD of the well containing the Avidity buffer and that containing the Wash Buffer is expressed as a percentage and gives the degree of avidity of the serum sample for the Rubella antigen.

4. KIT CONTENTS AND REAGENT PREPARATION:

- Reagents are sufficient for 24 determinations.
- **Bring reagents to room temperature before use.**

MT PLATE MICROPLATE COATED WITH RUBELLA VIRUS (COLOURED IN RED). 3 x 16 wells.

Use: open the package at the opposite end from the code (R followed by the lot number) which is useful for identification purposes, remove the support and wells to be used from the foil package, and place the unused wells in the polythene bag with the silica gel, expel the air and seal by pressing the closure.

CONJ CONJUGATE. 1x12 mL.

Contents: anti-human IgG monoclonal antibodies labelled with Peroxidase, in phosphate buffer solution containing phenol 0.05% and Bronidox 0.02%. Ready for use without further dilution.

CONTROL AVIDITY CONTROL (2.5 mL)

Contents: Human serum containing anti-Rubella IgG antibodies diluted in Phosphate buffer 0.01 mol/L with BSA 1% and sodium azide 0.09%, liquid, ready for use without further dilution.

AVIDITY BUF AVIDITY BUFFER (1 x 10 mL)

Contents: Urea in saline buffered solution containing phenol 0.05% and Bronidox 0.02%, ready for use. If crystals are present, they should be dissolved at 37°C before use.

WASH BUF 10x WASH BUFFER 10X. 1 x 100 mL. **INTERCHANGEABLE BETWEEN LOTS**

Contents: Phosphate buffered saline, concentrated 10 times; contains Brij 0.5% w/v.

Preparation: dilute 1:10 with distilled water in order to obtain the washing buffer ready for use. If crystals are present, they should be dissolved at 37°C before dilution.

SAMP DIL 50x DILUENT 50X. 1 x 4.5 mL. **INTERCHANGEABLE BETWEEN LOTS**

For dilution of serum samples.

Contents: Proteic solution conc. 50 times, containing BSA 5% w/v and Bronidox 0.02%.

Preparation: Dilute 1:50 in the washing buffer to obtain the diluent ready for use.

SUBS TMB SUBSTRATE. 12 mL. Ready for use. **INTERCHANGEABLE BETWEEN LOTS**

Contents: Tetramethylbenzidine and hydrogen peroxide stabilised in citrate buffer, pH 3.8.

H₂SO₄ 0.3 M STOP SOLUTION. 1x16 mL. **INTERCHANGEABLE BETWEEN LOTS**

H₂SO₄ 0.3 M, in solution ready for use.

ADHESIVE FILMS (2).

POLYTHENE BAG (1).

MATERIALS REQUIRED BUT NOT PROVIDED.

- Microplate reader, wavelength 450 nm or 450/620 nm, with OD linearity up to 2,000 (at least).
- Microplate washer (preferable) able to dispense volumes in the range 225-375 µL
- Distilled or deionized water
- Normal laboratory glassware: cylinders, test-tubes etc.
- Micropipettes for the accurate collection of 10, 100, 1000 µl solution
- Disposable gloves
- Timer
- Sodium Hypochlorite solution (5%)
- Containers for collection of potentially infectious materials
- Absorbent tissue.

5. STORAGE AND STABILITY OF REAGENTS

Reagents must be stored at 2/8°C.

The expiry date is printed on each component and on the box label.

Reagents have a limited stability after opening and/or preparation

REAGENT	CONDITIONS
Microplate	5 weeks at 2/8°C, polythene bag
Control serum	5 weeks at 2/8°C
Conjugate	5 weeks at 2/8°C
Substrate	up to the expiry date at 2/8°C, 1 week at 15-30°C; store in the dark
Sample Diluent	ready for use, 2 weeks at 2/8°C
Wash Buffer	2 weeks at 2/8°C, 5 days at 15/30°C.
Avidity Buffer	up to the expiry date at 2/8°C
Stop Solution	up to the expiry date at 2/8°C

6. PRECAUTIONS

FOR IN VITRO DIAGNOSTIC USE ONLY.

This kit contains materials of human origin which have been tested and gave a negative response by FDA-approved methods for the presence of HbsAg and for anti-HIV-1, anti-HIV-2 and anti-HCV antibodies. As no diagnostic test can offer a complete guarantee regarding the absence of infective agents, all material of human origin must be handled as potentially infectious. All precautions normally adopted in laboratory practice should be followed when handling material of human origin.

Health and Safety Information

1. Do not pipette by mouth. Wear disposable gloves and eye protection while handling specimens and performing the assay. Wash hands thoroughly when finished.
2. The following reagents contain low concentrations of harmful or irritant substances:
 - a) The Wash Buffer contains detergents
 - b) The conjugate contains phenol
 - c) The substrate is acid
 - d) The Control contains Sodium Azide 0.09% as preservative which may react with copper and lead in plumbing to form potentially explosive metal azides. On disposal, flush with large volumes of water to prevent azide buildup.

If any of the reagents come into contact with the skin or eyes, wash the area extensively with water.
3. Non-disposable apparatus should be sterilized after use. The preferred method is to autoclave for 1 h at 121°C; disposables should be autoclaved or incinerated.
4. Sulphuric acid required for the Stop Solution and hydrochloric acid used for washing glassware are corrosive and should be handled with appropriate care. If they come into contact with the skin or eyes, wash thoroughly with water.
5. Neutralized acids and other liquid waste should be decontaminated by adding a sufficient volume of sodium hypochlorite to obtain a final concentration of at least 1.0%. A 30 minute exposure to 1% sodium hypochlorite may be necessary to ensure effective decontamination.
6. Spillage of potentially infectious materials should be removed immediately with adsorbent paper tissue and the contaminated area swabbed with, for example, 1.0% sodium hypochlorite before work is continued. Sodium hypochlorite should not be used on acid-containing spills unless the spill area is first wiped dry. Materials used to clean spills, including gloves, should be disposed of as potentially biohazardous waste. Do not autoclave materials containing sodium hypochlorite.

Analytical precautions

1. Allow all reagents and samples to come to room temperature (18-30°C) before use. Immediately after use return reagents to the recommended storage temperature. **It is important to work at the correct temperature. Check that the thermostat does not go below 35°C or over 39°C.** Open the envelope containing the strips after at least ½ hr at room temperature.
2. Do not use the reagents beyond the stated expiry date. Microbiological contamination of reagents must be avoided as this may reduce the life of the product and cause erroneous results.
3. Do not modify the Test Procedure or substitute reagents from other manufacturers or other lots unless the reagent is stipulated as interchangeable. Do not reduce any of the recommended incubation times.
4. Any glassware to be used with the reagents should be thoroughly washed with 2M hydrochloric acid and then rinsed with distilled water or high quality deionized water.
5. Do not expose reagents to strong light or hypochlorite fumes during storage or during incubation steps.
6. Do not allow wells to become dry during the assay procedure.
7. Care must be taken not to cross-contaminate reagents. It is important that pipettes are dedicated for exclusive use with the various reagents.
8. Care should be taken to avoid touching or splashing the rim of the well with conjugate. Do not "blow-out" from microplates.
9. Enzyme immunoassays can occasionally exhibit an "edge effect" which must be minimized by increasing the humidity during incubation steps. Plates must be covered with their covers and incubated at 37°C either in a water bath with a rack or float to support the plates if necessary, or in an incubator. Alternatively, plates can be incubated in an approved analyzer. See the appropriate operating manual for further details. CO₂ incubators must not be used.
10. Ensure that the bottom of the plate is clean and dry, and that no bubbles are present on the surface of the liquid before reading the plate.
11. Use of highly hemolyzed samples, incompletely clotted sera, or samples with microbial contamination may give rise to erroneous results.
12. The use of the kit with automated instruments must be validated by the user.
13. For each instrument used, read the manufacturer's instructions manual carefully to obtain additional information on the following points:
 - installation and particular requisites
 - operating principles, instructions, precautions and risks
 - manufacturer's specifications and instrument performance
 - servicing and maintenance.
 -

7. TYPE AND STORAGE OF SAMPLE

The sample is composed of serum collected in the normal manner from the vein and handled with all precautions dictated by good laboratory practice. The fresh serum may be stored for 4 days at 2/8°C, or frozen for longer periods at -20°C; avoid repeated freeze/thaw cycles. Defrosted samples must be carefully shaken before performing the test.

O.D. with Avidity Buffer – O.D. blank
 _____ x 100 = % of avidity

O.D. with Wash Buffer – O.D. blank

A ratio over 45% indicates the presence of high avidity IgG antibodies; when it is lower than 35%, this indicates the presence of low avidity IgG. If the percentage is between 35 and 45%, there is a medium degree of avidity (borderline). The calculation can only be performed on samples which give an OD \geq 0.40 in the well washed with Wash Buffer.

WARNING: If the O.D. $<$ 0.4, repeat the test diluting the sample 1/101.

12. LIMITATIONS OF THE PROCEDURE

A result of high avidity does not exclude the possibility of a recent infection.

13. DIAGNOSTIC SENSITIVITY AND SPECIFICITY

115 samples were tested with the Diesse kit and in parallel with another commercial test; all the samples had been studied for the presence of specific IgG and IgM.

The samples were divided into the following groups:

- 1) 54 from subjects with a recent history of Rubella infection (44 positive in IgM, 4 doubtful and 6 negative).
- 2) 41 samples from pregnant women.
- 3) 20 samples from fertile women, positive for IgG and negative for IgM.

The Diesse kit identified 104 cases out of 115 with a correlation of 90% in comparison to the other method.

14. PRECISION

Table 1 Precision in run (n=6)

Sample	% average avidity	D.S.	CV%
Low avidity	14	1	8
High avidity	91	4	4

Tab.2 Precision between runs

Sample	% average avidity	D.S.	CV%
S1	71	5	7
S2	34	3	9
S3	6	2	35

14. TROUBLE SHOOTING GUIDE

PROBLEM	POSSIBLE SOURCE	TEST OR ACTION
Invalid run (all negative)	One or more reagents not added or added in wrong sequence	Recheck procedure Check for unused solutions. Repeat test.
	Unreactive plate	Check the code on the package containing the plate (see package insert point 4 for correct code).
		Check for moisture in unused plate. (Silica gel dessiccant must be pale yellow).Repeat test
Invalid run (all positive)	Contamination of substrate	Take new aliquot of substrate.
	Inadequate washing	Ensure that wash apparatus works well
Poor precision	Incomplete washing of wells	Ensure that wash apparatus works well
	Inadequate aspiration of wells	Ensure that wash apparatus works well
	Pipetting error	Check pipette function
	Reagent addition too slow	Avoid drying of the plate after washing step. Add reagents immediately
	Presence of bubbles	Avoid air bubbles during pipetting.
	Optical pathway not clean	Check instrument light source and detector for dirt. Wipe bottom of plate with soft tissue.
Inadequate Color development	Incorrect incubation times or	Check for temperature control and time

	temperature	monitoring
		Adhere to recommended instruction for use.
	Inadequate volume of substrate added to the plate	Check pipette function.

17. REFERENCES

1. S. Rousseau, K. Hedman. Rubella infection and reinfection distinguished by avidity of IgG. Lancet 1: 1108 (1988).
2. H. Thomas, C. Morgan. Rubella-specific IgG subclass avidity ELISA and its role in the differentiation between primary Rubella and rubella infection. Epidemiol. Infect. 101: 591 (1988).
3. L. Matter, K. Kogelschats, D. Germann. Serum levels of rubella virus antibodies indicating immunity: response to vaccination of subjects with low or undetectable antibody concentrations. J. Inf. Dis. (1997).

DIESSE Diagnostica Senese
Via delle Rose 10
53035 Monteriggioni (Siena) Italy.
Tel. 0577-58711