MASTAZYME TUBERCULOSIS

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11. Literatur

1. Bloom BR, Murray CJL. Tuberculosis: Commentary on a re-emergent killer. *Science*, **257**: 1055-64 (1992).
2. Kochi A. Global tuberculosis situation and the control strategy of the WHO. *Tubercle*, **72**: 1-6 (1991).
3. Marks LG. Genetics of tuberculosis. *Medical clinics of North America*, **77**(6): 1219-33 (1993).
4. Aziz A, Siddiqui SH, Ishaq M. Drug resistance of *Mycobacterium tuberculosis* from treated patients in Pakistan. *Tubercle*, **70**: 45-51 (1989).
5. Outbreak of multidrug-resistant tuberculosis Texas, California and Pennsylvania. *Morbidity and mortality weekly report*, **39**: 369-72 (1990).
6. TB morbidity United States, 1995. *Morbidity and mortality weekly report*, 1996, **45**: 365-70.
7. Barnes PF, Lee HQ, Davidson PT. Tuberculosis in patients with HIV infection. *Medical clinics of North America*, **77**(6): 1369-89 (1993).
8. Directorate-General for Chest Diseases. *Tuberculosis control guide*. Egypt, National Tuberculosis Control Programme, Ministry of Health, November 1994.
9. Snider DE Jr, La Montagne JR. The neglected global tuberculosis problem: A report of the 1992 World Congress on tuberculosis. *Journal of infectious diseases*, **169**: 1189-96 (1994).
10. Tuberculosis control as an integral part of primary health care. Geneva, World Health Organization, 1988.
11. Toman K. Tuberculosis case-finding and chemotherapy. Questions and answers. Geneva, World Health Organization, 3-74 (1979).

Contents	Page
1. Intended Use	10
2. Introduction	10
3. Principle of the Test	10
4. Kit Contents	11
5. Materials Required but not Provided	12
6. Warnings and Precautions	12
7. Storage and Stability	12
8. Specimen Collection and Handling	13
9. Assay Procedure	13
10. Results and Interpretation	14
11. Assay Performance	14
12. References	15

1. Intended Use

MASTAZYME TUBERCULOSIS (Mycobacterium tuberculosis) Antibody ELISA has been designed for the detection and the quantification of specific IgG / IgM / IgA antibodies respectively against Mycobacterium tuberculosis in serum and plasma. Further applications in other body fluids are possible and can be provided on request.

This assay is intended for *in-vitro* diagnostic use only.

All laboratory test results should be interpreted in conjunction with other clinical data. The clinical judgement and further tests have to be taken into account additionally.

2. Introduction

Mycobacterioses (tuberculosis, leprosy, atypical mycobacterioses, paratuberculosis and perhaps Crohn's Disease) are the infectious diseases of men and animals with the largest diffusion on earth. The infectious agents of tuberculosis are acid-resistant rod-like formed bacteria of the family Mycobacteriaceae, genus Mycobacterium. The germ was detected by Robert Koch in 1882. Owing to the very high infectious power of pathogenic mycobacteria, early diagnosis is essential to prevent spreading of the disease. Convergences of various approaches are necessary to control the mycobacterioses, immune reactions and bacterial shedding being variable during the diseases. However, usual diagnostic procedures were up to now unsatisfying and did not allow the differentiation of mycobacterial species.

The illness is normally transferred by droplets of saliva from infected persons. The target of the infection are mostly the lungs but also other organs like the brain, intestinal tract, bones, lymph nodes and kidneys can be afflicted. Tuberculosis is not only found in developing countries with 8 million of new infections yearly but also in industrialized civilisations as an actual disease with some thousands of cases yearly. Without treatment the disease leads in 50 % of the cases to death within less than two years. Clinical symptoms are fatigue, loss of weight, lack of appetite, light fever, nocturnal sweat and pain in the chest. Especially patients with HIV are threatened by tuberculosis due to their impaired immune system.

A vaccination with living attenuated bacteria is possible (BCG = Bacille Calmette Guérin). This is mostly done with newborn or young children. With older patients, before the vaccination there is normally performed the tuberculin test (Pirquet or Mantoux) where a small amount of tuberculin is injected under the skin. In a positive case there exist antibodies against Mycobacteria and a vaccination is not necessary.

Up to recently, there have not existed any serological methods to detect tuberculosis antibodies in serum. The only available procedure was besides the skin tuberculin test the direct microscopical identification of the dyed bacteria in sputum. Meanwhile specific antigens have been prepared either by purification of natural material or by recombinant methods. In the ELISA kits for the determination of IgG / IgM / IgA antibodies a cocktail of highly pure proteins is used in order to determine an immune response against the bacteria in human serum. A fresh or chronically active infection can be diagnosed by IgA and IgM tests.

3. Principle of the Test

The principle of the test reaction can be described in four stages.

3.1 Serum incubation

Specific antibodies bind to the antigens on the solid phase to form a stable immune complex. After a 60 minutes incubation at room temperature the wells are washed with prediluted wash buffer to remove all non-reactive serum components.

3.2 Conjugate incubation

The anti-human-IgG /-IgM /-IgA horseradish peroxidase conjugate is added to all wells. The conjugate binds to IgG / IgM / IgA antibodies on the solid phase antigen to form a stable sandwich. After a 30 minutes incubation at room temperature the excess conjugate is removed by washing all wells with washing buffer.

3.3 Substrate reaction and stopping

The TMB substrate is dispensed into each well and the peroxidase enzyme/substrate reaction forms a stable blue chromogen. The reaction and subsequently the colour development is stopped after 20 minutes incubation at room temperature by adding 0.5 N H₂SO₄ to the wells. The change in pH also causes the chromogen to change colour from blue to yellow.

3.4 Reading and interpretation

The intensity of the colour is read in a microtiter plate reader at 450 nm (recommended reference wavelength for bichromatic measurement: 600 - 690 nm). The intensity of the colour (OD) is directly proportional to the concentration of the specific antibody in the patient sample.

4. Kit Contents

The kits contains sufficient reagents for 12 x 8 = 96 determinations. The strips and solutions have to be stored at 4 - 8 °C. The expiry date is mentioned on the labels.

12	Microtiter strips	single strips each with 8 break-apart wells coated with the antigen of Mycobacterium tuberculosis
1 x	Frame holder	
4 x 2 mL	Calibrators 1 - 4	human serum containing antibodies against Mycobacterium tuberculosis (concentrations listed below) diluted in PBS and stabilised with 0.01 % methyliothiazolone and 0.01 % bromonitrodioxane as preservatives, ready to use

		IgG	IgM	IgA
Cal. 1 (negative)	Concentration (U/mL)	1	1	1
Cal. 2 (Cut-off)		10	10	10
Cal. 3 (weak positive)		40	30	30
Cal. 4 (positive)		150	100	100

1 x 60mL	Serum diluent	PBS/BSA buffer solution, contains < 0.1 % sodium azide as preservative, ready to use
1 x 12 mL	Enzyme conjugate solution	HRP-labelled goat anti-human-IgG /-IgM /-IgA, ready to use
1 x 12 mL	TMB substrate	3,3',5,5' Tetramethylbenzidine, ready to use
1 x 12 mL	Stopping solution	0.5 N sulfuric acid, ready to use
1 x 60 mL	Washing buffer 10 x concentrated	PBS/Tween buffer solution 10 x concentrated to be diluted 1:10 prior to use; the concentrate should be warmed up to 37 °C for 15 min to avoid any crystals
2 x	Plate sealers	to cover microtiter strips during incubation
1 x	Plastic bag	re-sealable for dry storage of non-used strips

5. Materials Required but not Provided

- 5 µL-, 100 µL- and 500-µL micro- and multichannel pipets
- Microtiter plate reader with a 450 nm filter (reference filter 600 – 690 nm)
- Microtiter Plate Washer (in case of manual washing: wash bottle)
- Reagent tubes for the serum dilution
- Measuring cylinder
- Distilled water or water of higher quality

6. Warning and Precautions

- For *in-vitro* diagnostic use only! Do not ingest or swallow! Laboratory safety precautions should be followed. Do not eat, drink or smoke in the laboratory.
- All sera and plasma or buffers based upon have been tested to HBsAg, HIV and HCV respectively with generally accepted methods and were found negative. Nevertheless, precautions like the use of latex gloves have to be taken.
- Serum and reagent spills have to be wiped off with a disinfecting solution (e.g. sodium hypochlorite 5 %) and have to be disposed of properly.
- All reagents have to be brought to room temperature (18 to 24 °C) before performing the test.
- Before pipetting all reagents should be mixed thoroughly by gentle tilting or swinging. Vigorous shaking with formation of foam should be avoided.
- It is important to pipet with constant intervals so that all the wells of the microtiter plate have the same conditions.
- When removing reagents out of the bottles care has to be taken that the stoppers are not contaminated. Further a possible mix-up has to be avoided. The content of the bottles is usually sensitive to oxidation so that they should be opened only for a short time.
- In order to avoid a carry-over or a cross-contamination separate disposable pipet tips have to be used.
- No reagents from different kit lots should be used and they should not be mixed with one another.
- All reagents have to be used within shelf life.
- In accordance with a Good Laboratory Practice (GLP) or following ISO 9001 all laboratory devices employed should be regularly checked regarding the accuracy and precision. This refers e.g. to microliter pipets and washing or reading (ELISA Reader) instrumentation.
- The contact of certain reagents especially the stopping solution and the substrate with skin, eye and mucosa has to be avoided because possible irritations and acid burns could arise and there exists a danger of intoxication.

7. Storage and Stability

Store all reagents at 4 - 8 °C.

The expiry date of each reagent is printed on the individual labels. Do not use any reagents after the expiry date has been exceeded.

The diluted washing buffer is stable for up to 4 weeks when stored at 4 - 8 °C.

8. Specimen Collection and Handling

Both serum or plasma (EDTA, heparin) can be used for the determination. Serum is separated from the blood which is aseptically drawn by venipuncture after clotting and centrifugation. The serum or plasma samples can be stored at 4 - 8 °C for up to 48 hours. They should be kept at -20°C for a longer storage. The samples should not be frozen and thawed repeatedly. Lipemic, hemolytic or bacterially contaminated samples can cause false positive or false negative results.

Patient sera must be prediluted **1:101 in serum diluent** (e.g. 5 µL serum + 500 µL serum diluent) prior to testing.

Samples containing concentrations higher than the highest calibrator have to be diluted further with serum diluent.

In case of interference with rheumatic factors, serum preabsorption with Rf absorbant (MASTSORB Order Code: 651003) is recommended. Do **not** absorb the calibrators.

9. Assay Procedure

9.1. Preparation of Reagents

Allow all kit components and specimens to reach room temperature (RT, 18 – 24 °C) prior to use and mix well.

Washing buffer: Dissolve any crystals which may be in the bottle by warming to 37 °C and then mix well.

Dilute the concentrated **washing buffer 1:10 with distilled water** (e.g. 60 mL buffer concentrate + 540 mL distilled water). Mix thoroughly.

- Strictly follow the instructions for reliable test performance. Any changes or modifications are within the responsibility of the user.
- All reagents and samples must be brought to room temperature before use, but should not be left at this temperature for longer than necessary.
- A standard curve should be established with each assay.
- Put the unused microtiter strips back in the plastic bag and store them dry at 4 - 8 °C.

9.2. Assay Steps

Prepare a sufficient amount of microtiter wells for calibrators, controls and samples.

Note: Other incubation conditions might be possible. In case of modifications of the recommended test procedure (e.g. incubation temperature 37 °C instead of RT) the user has to validate assay performance.

1. Pipette **100 µL** each of the **diluted (1:101) samples** and the **ready to use calibrators** into the appropriate wells.
2. Cover plate with the enclosed plate sealing foil and **incubate** at room temperature for **60 minutes**.
3. Discard the contents of the microwells and wash **3 times** with **300 µL** of **diluted washing buffer**. Afterwards remove residues of the washing solution by gentle tapping of the microtiter plate on a paper towel.
4. Pipette **100 µL** of **enzyme conjugate** solution into each well.
5. Cover plate with plate sealing foil and **incubate** for **30 minutes** at room temperature.
6. Discard the contents of the microwells and wash **3 times** with **300 µL** of **diluted washing buffer**. Afterwards remove residues of the washing solution by gentle tapping of the microtiter plate on a paper towel.
7. Dispense **100 µL** of **TMB substrate** into each well.

8. Cover plate with the plate sealing foil and **incubate** for **20 minutes** in the dark (e.g. drawer) at room temperature.
9. Add **100 µL** of **stopping solution** to each well.
10. After thorough mixing and wiping the bottom of the plate, **read the optical density** at **450 nm** and calculate the results. Blank against air. A bichromatic measurement using a reference wavelength of 600 - 690 nm is recommended.

The developed colour is stable for at least 60 minutes. Read optical densities during this time.

10. Results and Interpretation

Example

	OD 450 nm	corrected OD	Mean OD Value
Blank	0.012		
Calibrator 1 (negative)	0.024/0.026	0.004/0.006	0.005
Calibrator 2 (cut-off)	0.520/0.580	0.500/0.560	0.530
Calibrator 3 (weak positive)	1.100/1.132	1.080/1.112	1.096
Calibrator 4 (positive)	1.500/1.590	1.480/1.570	1.525

The table above should be considered as an example which was achieved under arbitrary temperature and environmental conditions. These data do NOT describe **reference values** which have to be found in other laboratories in the same way!

10.1. Qualitative Calculation

The calculated OD values for patient sera as mentioned above are compared with the value for the cut-off calibrator. If the value of the sample is higher, then it should be read as positive.

A value below the cut-off calibrator should be read as negative. It seems reasonable to define a range of $\pm 20\%$ around the value of the cut-off as a grey zone. It is recommended to repeat results laying within the grey zone using the same serum or a new sample of the same patient, taken after 2 - 4 weeks. Both samples should be measured in parallel in the same run.

The positive calibrator must show an absorption value at least double the value received by the cut-off calibrator.

10.2 Quantitative Calculation

The ready to use calibrators of the Mycobacterium tuberculosis antibody kit are defined and values expressed are in arbitrary units (U/mL). This gives access to an exact and reproducible quantification and in consequence patient antibody titer monitoring is possible. Concentration values for calibrators are printed on the labels of the vials.

A standard curve is plotted by entering the mean absorbance value of the calibrators on the Y-axis and the corresponding concentration on the X-axis using graph paper. The concentration of the patient samples can then be read directly from the graph.

The calculation of the result can be performed using a computer and a suitable software program.

11. Assay Performance

The assay characteristics of the MASTAZYME TUBERCULOSIS IgG / IgM / IgA ELISAs have been established and assessed according the European IVD directive. Detailed validation data can be provided on special request.

12. References

1. Bloom BR, Murray CJL. Tuberculosis: Commentary on a reemergent killer. *Science*, **257**: 1055-64 (1992).
2. Kochi A. Global tuberculosis situation and the control strategy of the WHO. *Tubercle*, **72**: 1-6 (1991).
3. Marks LG. Genetics of tuberculosis. *Medical clinics of North America*, **77**(6): 1219-33 (1993).
4. Aziz A, Siddiqui SH, Ishaq M. Drug resistance of *Mycobacterium tuberculosis* from treated patients in Pakistan. *Tubercle*, **70**: 45-51 (1989).
5. Outbreak of multidrug-resistant tuberculosis Texas, California and Pennsylvania. *Morbidity and mortality weekly report*, **39**: 369-72 (1990).
6. TB morbidity United States, 1995. *Morbidity and mortality weekly report*, 1996, **45**:365-70.
7. Barnes PF, Lee HQ, Davidson PT. Tuberculosis in patients with HIV infection. *Medical clinics of North America*, **77**(6): 1369-89 (1993).
8. Directorate-General for Chest Diseases. *Tuberculosis control guide*. Egypt, National Tuberculosis Control Programme, Ministry of Health, November 1994.
9. Snider DE Jr, La Montagne JR. The neglected global tuberculosis problem: A report of the 1992 World Congress on tuberculosis. *Journal of infectious diseases*, **169**: 1189-96 (1994).
10. Tuberculosis control as an integral part of primary health care. Geneva, World Health Organization, 1988.
11. Toman K. *Tuberculosis case-finding and chemotherapy. Questions and answers*. Geneva, World Health Organization, 3-74 (1979).

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Laboratoriums-Präparate GmbH
MAST GROUP Ltd.