MASTAFLUOR™ FTA-ABS Sorbent Set

MASTAFLUOR™ FTA-ABS Sorbent Set

REF 626281SO

**Gebrauchsinformation /
Instructions for Use /
Notice Technique**

**Nur zur Verwendung durch Fachpersonal /
For professional use only /
Exclusivemen à un usage professionnel**

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MASTAFLUOR™ FTA-ABS Sorbent Set

Intended Use

MASTAFLUOR™ FTA-ABS Sorbent is intended to be used for the absorption of non-specific anti-Tp-antibodies together with the other MASTAFLUOR™ FTA-ABS kit reagents.

FOR IN VITRO DIAGNOSTIC USE ONLY

Important note for use of these kit instructions

Any assay relevant changes to the kit instructions for use (IFU) will lead to a change of the version number at the bottom on the last page of the IFU. All changes made will be identified on a separate sheet added to the IFU for a period of three months from the date of a change of the version. Please ensure that the latest version of the IFU is used for the assay procedure.

Principle of the Test

See MASTAFLUOR™ FTA-ABS kit instruction.

Kit Contents

1. MASTAFLUOR™ FTA-ABS Sorbent (Ultrasonicate)

2 bottles containing culture extract of *Treponema reiteri*, ready to use (4 mL each). Contains 0.1 % sodium azide as preservative.

2. Instruction Leaflet

Materials Required but not Provided

1. MASTAFLUOR™ FTA-ABS (REF 630522).

Warning and Precautions

1. The reagents supplied in this kit are for in vitro diagnostic use only.
2. Read instructions carefully before conducting the assay. Do not modify the procedure without validation.
3. Do not use kit beyond the expiry date.
4. Wear appropriate protective clothing and use appropriate facilities.
5. Do not mouth pipette.

6. Use disposable plastic ware where ever possible. Re-usable glassware should be washed thoroughly and rinsed free of detergents before use.
7. Do not mix reagents between different lots, as reagents have been calibrated for each batch.
8. Do not cross-contaminate reagents or interchange caps on bottles. Use a separate pipette or pipette tips for each sample and reagent.
9. Sodium azide is used as a preservative as marked. It may be toxic if ingested. Sodium azide may react with lead and copper plumbing to form highly explosive salts. Always dispose by flushing to drain with plenty of water.

Stability and Storage

The MASTAFLUOR™ FTA-ABS Sorbent (Ultrasonicate) may be stored at 2-8 °C for up to at least 6 month.

Test Procedure

1. Allow all materials to reach room temperature (at least 20 °C) prior to use; mix reagents thoroughly before use.
2. For testing follow kit instruction MASTAFLUOR™ FTA-ABS.

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