



Mast Group Ltd. Mast House, Derby Road, Bootle, Merseyside, L20 1EA United Kingdom

Tel: + 44 (0) 151 472 1444 Fax: + 44 (0) 151 944 1332 email: sales@mast-group.com Web: www.mast-group.com



Mast Diagnostica GmbH Feldstrasse 20 DE-23858 Reinfeld

Tel: + 49 (0) 4533 2007 0 Fax: + 49 (0) 4533 2007 68 email: mast@mast-diagnostica.de Web: www.mast-group.com



12 rue Jean-Jacques Mention CS91106, 80011 Amiens, CEDEX 1 France

Tél: + 33 (0) 3 22 80 80 67 Fax: + 33 (0) 3 22 80 99 22 email: info@mast-diagnostic.fr Web: www.mast-group.com



MASTDISCS® Combi ESβL Detection Set (EUCAST)

D76C

Intended use

For the detection of extended spectrum beta-lactamases (ES\(\mathbb{L} \mathbb{S})\) in Enterobacterales.

FOR IN VITRO DIAGNOSTIC USE ONLY

Contents and Formulation*

2 paired sets of cartridges per pack, each cartridge containing approximately 50 discs:

Set 1 CTX5 Cefotaxime 5 μg discs (x1)

CTXCV Cefotaxime 5 µg + clavulanic acid 10

ug discs (x1)

Set 2 CAZ10 Ceftazidime 10 μg discs (x1)

CAZCV Ceftazidime 10 μg + clavulanic acid

10μg discs (x1)

Storage and shelf life

Store at 2 to 8°C in the containers provided until the expiry date shown on the pack label. Allow to equilibrate to room temperature before opening.

Precautions

For *in vitro* diagnostic use only. Observe approved biohazard precautions and aseptic techniques. To be used only by adequately trained and qualified laboratory personnel. Sterilise all biohazard waste before disposal. Refer to Product Safety Data sheet.

Materials required but not provided

Standard microbiological supplies and equipment such as loops, MAST® culture media, Mueller-Hinton agar, swabs, forceps, callipers etc., as well as an incubator capable of maintaining $35 \pm 1^{\circ}$ C.

Procedure

- Using a pure, fresh culture of the test organism, prepare a suspension equivalent in density to a 0.5 McFarland standard in physiological saline.
- Using a sterile swab, spread the suspension uniformly across the surface of a single Mueller Hinton Agar plate in accordance with the European Committee on Antimicrobial Susceptibility Testing (EUCAST) procedure.
- 3. Using a MAST® DISCMASTER Dispenser, or alternatively a sterile needle or forceps, place one of each disc onto the plate of inoculated medium, ensuring sufficient space between the discs to allow formation of clearly defined zones of inhibition.
- 4. Incubate at 35 ± 1 °C for 18 ± 2 hours.
- Measure and record the diameter of any zones of inhibition, to the nearest whole millimetre. Discs showing no zone of inhibition should be recorded as 6 mm.

Interpretation of results

Compare the zone of inhibition for the cephalosporin disc to that of the corresponding cephalosporin plus clavulanic acid combination disc. An increase in zone diameter of $\geq 5 \text{mm}$ in the presence of clavulanic acid for one or both of the sets indicates the presence of ES βL in the test organism.

Quality control

Check for signs of deterioration. Quality control must be performed with at least one organism to demonstrate a positive reaction and at least one organism to demonstrate a negative reaction. Zones of inhibition obtained using combination disc plus clavulanic acid and corresponding cephalosporin only disc against ES β L-negative control organism *E. coli.* ATCC® 25922 should be equal or show no greater difference in diameter than ± 2 mm. Any greater difference implies malfunction or deterioration. Do not use the product if the reactions with the control organisms are incorrect. The list below illustrates a range of performance control strains which the end user can easily obtain:

Test Organism	Result
Escherichia coli	Positive
NCTC 13351	
Escherichia coli	Positive
NCTC 13353	
Escherichia coli	ES βŁ Negative
ATCC® 25922	

Limitations

D76C is not suitable for testing *Pseudomonas* spp. or *Acinetobacter* spp. To avoid potentially erroneous results do not mix cartridges from different batches and ensure all discs in the set are tested on the same plate.

References

Bibliography available on request.