

Mast Group Ltd.

Mast House, Derby Road, Bootle, Merseyside, L20 1EA United Kingdom

Tel: + 44 (0) 151 472 1444 Fax: + 44 (0) 151 944 1332 email: sales@mast-group.com Web: www.mast-group.com



Mast Diagnostica GmbH Feldstrasse 20 DE-23858 Reinfeld Germany

Tel: + 49 (0) 4533 2007 0 Fax: + 49 (0) 4533 2007 68 email: mast@mast-diagnostica.de Web: www.mast-group.com



12 rue Jean-Jacques Mention CS91106, 80011 Amiens, CEDEX 1 France

Tél: + 33 (0) 3 22 80 80 67 Fax: + 33 (0) 3 22 80 99 22 email: info@mast-diagnostic.fr Web: www.mast-group.com



MASTDISCS® ID Carbapenemase Activity Test (CAT-ID) discs

D71C

Intended use

For the detection of carbapenemase activity in Enterobacterales.

FOR IN VITRO DIAGNOSTIC USE ONLY

Contents

5 cartridges per pack, each cartridge containing approximately 50 discs.

Formulation*

6mm diameter filter paper discs impregnated with faropenem.

Storage and shelf life

Store at 2 to 8°C in the containers provided until the expiry date shown on the pack label. Allow to equilibrate to room temperature before opening.

Precautions

For in vitro diagnostic use only. Observe approved biohazard precautions and aseptic techniques. To be used only by adequately trained and qualified laboratory personnel. Sterilise all biohazard waste before disposal. Refer to Product Safety Data sheet.

Materials required but not provided

Standard microbiological supplies and equipment such as loops, MAST® culture media, Mueller-Hinton agar, swabs, forceps etc., as well as an incubator capable of maintaining $35 \pm 1^{\circ}$ C.

Procedure

- 1. Using a pure, fresh culture of the test organism, prepare a suspension equivalent in density to a 0.5 McFarland standard in physiological saline.
- 2. Using a sterile swab, spread the suspension uniformly across the surface of a Mueller Hinton Agar plate in accordance with the European Committee on Antimicrobial Susceptibility Testing (EUCAST) procedure.
- 3. Using a MAST® DISCMASTER Dispenser, or alternatively a sterile needle or forceps, place a disc on to the plate of inoculated medium.
- 4. Incubate at $35 \pm 1^{\circ}$ C for 18 ± 2 hours.
- 5. Record the presence or absence of a zone of inhibition around the disc, and note the presence of any microcolonies within the zone.

Interpretation of results

No zone of inhibition is indicative of carbapenemase activity e.g. MβL (metallo-β-lactamase) or KPC (Klebsiella pneumoniae carbapenemase). A small number of OXA-48 (oxacillinase) producing organisms may also appear resistant.

A clearly defined zone of inhibition is indicative of no carbapenemase activity.

A zone of inhibition with colonies growing within the zone (double zone) is indicative of OXA-48 (oxacillinase) production.

Quality control

Check for signs of deterioration. Quality control must be performed with at least one organism to demonstrate a positive reaction and at least one organism to demonstrate a negative reaction. Do not use the product if the reactions with the control organisms are incorrect. The list below illustrates a range of performance control strains which the end user can easily obtain:

Test Organism	Result
Klebsiella pneumoniae	No zone of inhibition
NCTC 13440 (MβL)	
Klebsiella pneumoniae	No zone of inhibition
NCTC 13438 (KPC)	
Klebsiella pneumoniae	Double zone
NCTC 13442 (OXA-48)	
Escherichia coli	Clear zone of
ATCC® 25922 (Negative)	inhibition

Limitations

D71C is not suitable for use with Pseudomonas spp. or Acinetobacter spp.

The presence of a carbapenemase resistance mechanism should be confirmed by phenotypic (e.g. MASTDISCS® Combi Carba plus: D73C) or genotypic methods. A small proportion of non-carbapenemase producing Enterobacterales may demonstrate resistance to faropenem.

References

Bibliography available on request.