self
LoliSponge™

Instructions for Use

CE

Saliva collection device Instructions for use

INTENDED USE

Copan LoliSponge™ is a saliva specimen collection system.

SUMMARY AND PRINCIPLES

Infections can be diagnosed on saliva specimens (1,2,3,4,5,6) by means of molecular biology techniques. For this purpose, saliva can be collected using the Copan LoliSponge™ device.

STORAGE


Store in its original package at a temperature between 2 and 25°C until the time of use. Do not freeze prior to use.

SPECIMEN COLLECTION, STORAGE AND TRANSPORT

It is advisable to keep the specimen refrigerated in order to best preserve its quality. Where not possible, specimens can be transported at room temperature and once in the laboratory stored at refrigerated temperature (2-8°C) until the time of processing. In any event, refer to the specimen transport and storage conditions declared in the methodology of the test used downstream.

CONTENT

Copan LoliSponge™ is a device consisting of a labelled plastic test tube with screw cap. The cap features a stick with absorbent sponges. Copan LoliSponge™ is supplied in a transparent bag.

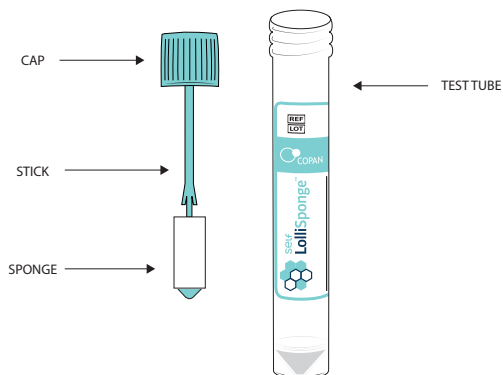MATERIALS REQUIRED BUT NOT INCLUDED

Laboratory centrifuge, micropipettes and tips, materials for extraction and analysis, materials for specimen freezing, gloves and single-use laboratory devices.

PRODUCT CODE LIST

Catalogue No.	Product description	Packaging	Sampling sites
1E063N01	Single bag containing a single-use device consisting of a plastic test tube with screw cap. Inside the test tube, the cap features a plastic stick with sponge.	30 individually packed devices 30 x 10 devices per box	Mouth

INSTRUCTIONS FOR USE AT THE COLLECTION SITE

Self-collection of saliva specimens by the patient.

NOTES: Carefully read the instructions before collecting the specimen.

Preferably collect the saliva in the morning (7).

1. Before starting, wash your hands with soap and water, rinse and dry them or sanitize them.
2. Open the individual bag and take out the LoliSponge™ test tube.
3. Open the LoliSponge™ test tube holding the device by the cap and insert it in your mouth. Gently move it around in your mouth (top of the tongue and between the teeth and cheeks) for 60 seconds so that the sponges are well moistened.
4. Place the cap on the LoliSponge™ test tube, screw it on and securely close the test tube.
5. Return the LoliSponge™ device following the instructions provided separately.

INSTRUCTIONS FOR USE FOR PROCESSING IN THE LABORATORY FOR MOLECULAR ANALYSIS

The analytical results depend on the correct collection, transport and storage conditions as well as the relative processing in the laboratory. In using molecular methods, take the necessary precautions to prevent cross-contamination. Separation of workspaces and a unidirectional workflow are essential to prevent cross-contamination.

1. Following specimen collection, the device must be centrifuged in order to extract the saliva from the sponge. The recommended protocol is 450G for 60 seconds.
NOTES: a. Do not exceed in time and speed in order to prevent detachment of the sponges and the impossibility to process the specimen.
b. Take care to hold the device in vertical position after centrifugation in order to prevent re-absorption of the saliva.
2. Take the device and unscrew the cap with the attached stick and sponges.
3. Using a micropipette, remix the saliva specimen that has deposited on the bottom of the test tube in order to homogenize it to dissolve any pellets that have formed due to centrifugation and aliquot it into dedicated extraction test tubes.
4. Proceed with specimen extraction and amplification of the desired molecular target.

QUALITY CONTROL

The device is regularly tested for the absence of cytotoxicity on the sponge batches intended for the product. The device is tested for the microbial load present. The final device is tested for the absence of interference on the molecular chemistries.

LIMITATIONS

1. In the laboratory, wear protective gloves and other universally recognized protections and precautions when handling clinical specimens. Observe the Biosafety recommendations when handling or analysing patient specimens.
2. The conditions, time and volume of specimens collected for clinical analysis are significant variables in obtaining reliable results. Follow the recommended guidelines for specimen collection.
3. Use is limited to molecular biology; not for cultural use or micro- and macroscopic examinations.
4. The product is non-sterile. The production materials and processes have been subjected to strict controls in order to guarantee high standards of cleanliness and quality for saliva collection purposes.
5. The product is not suitable for the collection and analysis of human DNA from saliva.
6. The product is not compatible with microbiome analysis.

WARNINGS

Carefully follow the instructions for use.

1. This product is intended for single use; reuse may cause a risk of infection and/or unreliable results.
2. The device is ready for use and does not require any preparation for specimen collection.
3. Not suitable for any application other than its intended use.
4. Do not repackage.
5. Do not force or excessively press LolliSponge™ during collection, as this may result in accidental breakage of the stick or its detachment from the cap.
6. Do not detach the stick with the sponge from the cap.
7. Do not bite the sponges.
8. The use of this product together with any rapid diagnostic kit or any diagnostic instrument is to be evaluated by the user prior to use.
9. Do not use if:
 - LolliSponge™ is visibly damaged (i.e. if the swab stick or sponges are broken);
 - if the package or test tube is open;
 - if the device shows signs of damage, deterioration, contamination or if it has passed the expiration date.
10. Do not touch the sponge with your hands before collection.
11. Do not spit the saliva directly into the test tube.
12. Collect the saliva at least 30 minutes after eating, drinking, smoking or brushing your teeth.
13. It must be assumed that all specimens contain infectious microorganisms; therefore, all specimens must be handled applying the appropriate precautions. After use, dispose of the test tubes and the swabs according to the laboratory regulations for infectious waste.
14. After specimen centrifugation, hold the test tube straight up in order to prevent the sponge from re-absorbing the specimen.

PERFORMANCE CHARACTERISTICS

The device has been validated by means of a molecular test for the search for SARS-CoV-2; the collected saliva was processed as indicated by Vogels et al. (2), making a change to the thermal amplification profile: 5' at 95°C followed by 5' at 4°C. 5 µl of the saliva treated with proteinase K and heat inactivated were amplified in qPCR (Applied Biosystems 7500 Fast) using N1 (FAM, BHQ-1, marked probe) and RP (Cy5, BHQ-1) according to what has been published by Centers for Disease Control (6). The stability of the saliva specimen on the LolliSponge™ device has been tested at time zero and after 3 days at room temperature using the above described protocol.

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BIBLIOGRAPHY

1. A.L. Wyllie et al. Saliva or nasopharyngeal swab specimens for detection of SARS-CoV-2, N. Engl. J. Med. 383(2020) 1283–1286.
2. C.B.F. Vogels et al. SalivaDirect: A simplified and flexible platform to enhance SARS-CoV-2 testing capacity, Med2, 263–280, March 12, (2021).
3. I. Yokota et al. Mass screening of asymptomatic persons for SARS-CoV-2 using saliva, Clin. Infect. Dis. (2020).
4. S. Iwasaki et al. Comparison of SARS-CoV-2 detection in nasopharyngeal swab and saliva, J. Infect. 81 (2020), e145–e147.
5. M. Riccò et al. RT-qPCR assays based on saliva rather than on nasopharyngeal swabs are possible but should be interpreted with caution: results from a systematic review and metaanalysis, Acta Biomed. 91 (2020) 1–15.
6. E. Borghi et al Pharmacological Research, Volume 165, March 2021, 105380.
7. D L-L Hung et al. Early-Morning vs Spot Posterior Oropharyngeal Saliva for Diagnosis of SARS-CoV-2 Infection: Implication of Timing of Specimen Collection for Community-Wide Screening Open Forum Infectious Diseases Volume 7, Issue 6, June 2020, ofaa210.

INDEX OF SYMBOLS

Symbol / Simbolo / Símbolo / Symbol / Symbole / Símbolos / Simbool	Meaning / Significato / Signification / Bedeutung / Sens / Significado / Betekenis
	Manufacturer / Fabricante / Fabricante / Hersteller / Fabricant / Fabricante / Fabrikant
	CE marking / Marchio CE / Marca CE / CE-Kennzeichnung / Marquage CE / Marcação CE / CE-merk
	Do not reuse / Non riutilizzare / No reutilizar / Nicht zur Wiederverwendung / Ne pas réutiliser / Não voltar a utilizar / Niet opnieuw gebruiken
	Catalogue number / Numero di catalogo / Número de catálogo / Bestellnummer / Référence du catalogue / Número do catálogo / Catalogusnummer
	Temperature limitation / Limiti di temperatura / Limites de temperatura / Temperature Begrenzung / Limites de temperature / Limites de temperatura / Temperatuurgrenzen
	Use by / Utilizzare entro / Fecha de caducidad / Verwendbar bis / Utiliser jusque / Prazo de validade / Te gebruiken vóór
	Consult Instructions for Use / Consultare le istruzioni per l'uso / Consulte las instrucciones de uso / Gebrauchsanweisung beachten / Consulter les instructions d'utilisation / Consultar as instruções de utilização / De gebruiksaanwijzing raadplegen
	Batch code (Lot) / Codice del lotto (partita) / Código de lote (Lote) / Chargencode (Chagenbezeichnung) / Code de lot (Lot) / Código do lote (Lote) / Partijcode
	Contains sufficient for <n> tests / Contenido / Contenuto sufficiente per <n> test / Contenido suficiente para <n> pruebas / Ausreichend für <n> Tests / Contenu suffisant pour <n> tests / Contém o suficiente para <n> testes / Voldoende inhoud voor <n> test
	Do not use if package is damaged / Non utilizzare in caso di confezionamento danneggiato / No utilizar en caso de paquete dañado / Bei Beschädigung der Verpackung nicht verwenden / Ne pas utiliser si l'emballage est abîmé / Não utilizar se a embalagem estiver danificada / Niet gebruiken in geval van beschadigde verpakking
	Non-sterile / Non sterile / Non stérile / Nicht steril / No estéril / Não estéril / Niet steriel

Copan Italia S.p.A.
Via F. Perotti, 10
25125 Brescia Italy
Tel +39 030 2687211
Fax +39 030 2687250

Email: info@copangroup.com
Website: www.copangroup.com

Innovating Together™

CHANGE HISTORY

Latest Revision N.	Issue date	Change History
00	2021-04	First Issue