INSTRUCTIONS FOR USE FOR THE USER

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Before proceeding read carefully the precautions, warnings and instructions for use reported on both sides of this document

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PRECAUTIONS

Open the swab pouch just before use and be careful not to touch any surfaces with the swab tip. After completing the procedure store the tube at room temperature. Send sample following the delivery instructions supplied separately by the kit provider. In case of self-collection on minor, use the product in presence of an adult.

WARNINGS

Do NOT reopen the tube after the liquid has been released from the cap. Wash hands thoroughly after handling. Avoid contact with eyes, skin or mucous membranes. If contact does occur, immediately wash with plenty of water. Not for Use in Diagnostic Procedures. Keep out of reach of children. Safety data sheet (MSDS) available on request by emailing at info@copangroup.com

IMPORTANT NOTICE

This product is supplied by Copan with certain Labeling, Instructions For Use and Hazard Pictograms. Buyer is committed to promote, offer and monitor the product sale through its supply chain with appropriate Labeling, Instructions For Use and Hazard Pictograms, in compliance with the legal and regulatory requirements applicable in the final market of the product, especially in case the assigned intended use differs from or exceeds the original product intended use declared by Copan.

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(EN) Danger – Contains Guanidine Thiocyanate - Harmful if swallowed - Causes skin irritation - Causes serious eye damage - Harmful to aquatic life with long lasting effects
Wash hands thoroughly after handling - Wear protective gloves/protective clothing/eye protection/face protection - Avoid release to the environment - IF ON SKIN: Wash with plenty of water - IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses if present and easy to do – continue rinsing. Immediately call a POISON CENTER/doctor - Dispose of contents/container in accordance with regulation

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