## MASTAZYME™ Chlamydia Transportmedium

MASTAZYME™ Chlamydia Transportmedium

**REF** 695020

95 x 1 mL

UDI-DI: 4250729700187

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## MASTAZYME™ Chlamydia Transport Medium

### Intended Purpose

The MASTAZYME™ Chlamydia Transport Medium is intended for transport of endocervical and urethral swab samples and for sample preparation in the MASTAZYME™ Chlamydia ELISA.

The MASTAZYME™ Chlamydia Transport Medium is to be used only by trained personnel in appropriately equipped laboratories and in compliance with laboratory safety guidelines.

### Important Note for Use of these Kit Instructions

Any assay relevant changes to the kit instructions for use (IFU) will lead to a change of the version number at the bottom on the last page. All changes made will be identified on a separate sheet added to the IFU for a period of three months from the date of a change of the version. Please ensure that the latest version of the IFU is used for the assay procedure.

### Principle of the Test

The MASTAZYME™ Chlamydia Transport Medium is ready-to-use and requires no further preparation.

After specimen collection and mixing, the specimen can be transported in the tightly closed specimen transport tube.

### Content

#### Chlamydia Transport Medium

95 x 1 mL transport medium in pre-filled screwable specimen transport tubes, ready-to-use.

### Abbreviation used

1. RTU ready to use

### Materials Required but not Provided

1. MASTAZYME™ Chlamydia Swab Set (Art. No.: 695030, UDI-DI: 05060488970986). Only swabs made of Dacron may be used for urogenital and endocervical specimen collection. The swabs must not have a wooden shaft. Swabs with transport medium of calcium alginate, agar or charcoal must not be used (Dacron is a registered trade name of DuPont Inc.).

2. For further detection of *Chlamydia trachomatis*, e.g.: MASTAZYME™ Chlamydia ELISA (Item No.: 695010, UDI-DI: 4250729700170) for the detection of Chlamydia antigen in urogenital specimens using an enzyme immunoassay to aid in the diagnosis of chlamydiosis.

### Warnings

1. Comply with the general health and safety guidelines for working with potentially infectious materials. Wear appropriate protective clothing and use appropriate lab facilities.
2. Do not use contaminated reagents. Do not use reagents with damaged bottles or packaging. Do not use reagents that show colour changes or turbidity.
3. Handle all specimens carefully and avoid cross-contamination of specimen containers.
4. The kit reagents may contain bovine components. To protect against infections with prions or zoonotic pathogens, standard occupational safety regulations (GLP) must be observed.

### Precautions / Notes

1. Read instructions carefully before conducting the assay. Do not modify the procedure without prior validation.
2. Do not use reagents beyond the expiry date.
3. The MASTAZYME™ Chlamydia Transport Medium is intended for the transport of urogenital swabs. All safety regulations relevant to the handling of infectious material must be observed. Dispose of all specimens after use according to local safety regulations for potentially infectious material.
4. The MASTAZYME™ Chlamydia Transport Medium is validated with the MASTAZYME™ Chlamydia Kit (Part No.: 695010, UDI-DI: 4250729700170) (Lit.1,2). Use of this product in combination with other diagnostic kits or instruments must be validated by the user prior to use.

### Disposal

Kit reagents, samples and contaminated disposables should be disposed of in accordance with the relevant disposal guidelines and regulations for infectious materials.

### Storage and Stability

Clinical swab specimens can be stored in transport medium at 2-8 °C for up to 8 days. For longer storage up to 6 months, specimens should be frozen at -80 °C. However, repeated thawing and freezing cycles should be avoided.

## Preparation for testing

The MASTAZYME™ Chlamydia Transport Medium is ready-to-use and requires no further preparation.

## Test procedure

1. Collect the specimen using the swab (MASTAZYME™ Chlamydia Swab Set) as described in the Specimen Collection section.
2. Dip the tip of the swab into the transport medium, carefully break off the handle at the notch, close the container and then shake it vigorously.
3. Send the closed container to the appropriate laboratory according to the specifications for transport (see section Storage and shelf life).

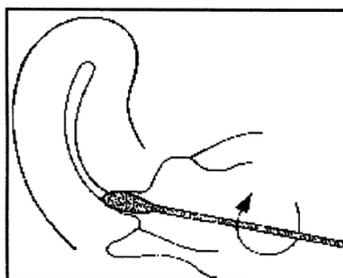
## Collection of Specimens

Specimens from the female cervix and urethra and from the male urethra should contain as many epithelial cells as possible.

If samples are to be additionally tested for gonorrhoea, this should be done with a special swab before taking the chlamydia test.

## Female endocervical specimens


1. Clean the exocervical area with sterile gauze or swab before sampling, to remove excess mucus.
2. Remove the swab from the packaging. Do not touch the applicator in the area below the break-off line.
3. Insert the appropriated swab into the endocervical canal and rotate vigorously for 5–10 seconds.

4. Move the swab to the portio region and rotate vigorously, then withdraw the swab without touching the vaginal walls.
5. For urethral swabs (approximately 30 % of infections), insert the swab 2-3 cm into the urethra while turning. Rotate the swab vigorously for about 10 seconds to ensure contact with the cylindrical epithelium.
6. Dip the tip of the swab into the transport medium, carefully break off the handle at the notch, close the container and then shake it vigorously.
7. Send the closed tube with swab and transport medium to the appropriate laboratory.
8. Dispose of the part of the swab handle that broke off at the top.

## Male urethral specimens

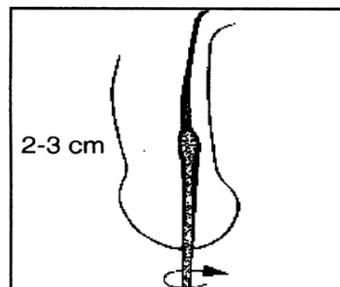1. Specimens should not be collected if the male has urinated within the previous hours.
2. Remove the swab from the packaging. Do not touch the applicator in the area below the break-off line.
3. Insert the appropriate male swab 2–3 cm into the urethra rotating the swab for 10 seconds and ensuring that all surfaces of the urethra are contacted.

4. Withdraw the swab and immerse into MASTAZYME™ Chlamydia Transport Medium, carefully break off the plastic shaft at the notch and then shake it vigorously.
5. Send the closed container with the swab and transport medium to the appropriate laboratory.
6. Dispose of the part of the swab handle that broke off at the top.

## Quality Control

All Lots of MASTAZYME™ Chlamydia Transport Medium are tested for microbial contamination by an external laboratory.

## Reporting serious incidents

All serious incidents that have occurred in connection with the device must be reported to the manufacturer and the competent authority of the Member State in which the user and / or patient is established.

## References

The references can be found at the end of the instructions for use.

## Referenzen / References:

1. Clark A., Stamm W.E., et al., Multicenter Evaluation of the AntigEnz Chlamydia Enzyme Immunoassay for Diagnosis of *Chlamydia trachomatis* Genital Infection. *Journal of Clinical Microbiology* 1992; 30 (11): 2762-2764
2. Mumtaz G., Clark S. et al., Comparison of an enzyme immuno assay (Antigenz Chlamydia) with cell culture for the detection of genital chlamydial infection in high and low risk populations. *Genitourin Med* 1993; 69: 119-122

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Change history includes changes to significant aspects of the assay / IFU.

Chapter	Description of change
Warnings and Precautions	Separation into two different sections Inclusion of additional points for assay handling
Disposal	Separation from warnings and precautions

**Mast Diagnostica GmbH,**

Feldstraße 20,  
23858 Reinfeld,  
Deutschland

Tel: +49 (0)4533 2007 0  
Fax: +49 (0)4533 2007 68  
email: mast@mast-diagnostica.de  
Web: www.mast-group.com

**Mast Group Ltd.**

Mast House, Derby Road  
Bootle, Merseyside, L20 1EA  
United Kingdom

Tel: +44 (0)151 472 1444  
Fax: +44 (0)151 944 1332  
email: sales@mastgrp.com  
Web: www.mast-group.com

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**Icons are used according to DIN EN ISO 15223-1**