Loop Cleansing Solution

CE 

Loop Cleansing Solution

ITALIANO	3
FRANÇAIS	4
ENGLISH	5
中文.....	6
DEUTSCH	7
ESPAÑOL	8
PORTUGUÊS	9
SRPSKA	10
TÜRK	11
한국어	12
日本人	13
ČESKY	14
ROMÂNĂ	15
DANSK	16
MAGYAR	17
NORSK	18
РУССКИЙ	19
SVENSKA	20
EESTI	21
LATVIEŠU	22
LIETUVIŲ	23
SLOVENSKÉ	24
УКРАЇНСЬКА	25

Copan

Copan Wasp S.r.l.
Via Achille Grandi, 32
25125 Brescia - Italy
Tel: +39 030 2687211/218
Fax: +39 0302659932

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Loop Cleansing Solution FOR AUTOMATIC CLEANING OF LOOPS AND SPREADERS

ENGLISH

INTENDED PURPOSE

For automatic cleaning of loops and spreaders.

INTENDED USE

The **Loop Cleansing Solution** is a washing solution intended to be used for the removal of solid residues possibly present on Copan WASP loops and spreaders. The **Loop Cleansing Solution** is used during the automatic wash cycle performed by Copan WASP instruments when required.

PRODUCT DESCRIPTION

The **Loop Cleansing Solution** is available in two formats indicated in the table below:

REF	Name	Description	Package
WR086-03-0005	Loop Cleansing Solution 5 ml	5 ml of Cleansing Solution in 12x80 mm polypropylene tube with black screw-cap	6 units of 50 tubes
WR086-03-0019	Loop Cleansing Solution 10 ml	10 ml of Cleansing Solution in 16x100 mm polypropylene tube with green screw-cap	6 units of 50 tubes

INSTRUCTIONS FOR USE

1. Remove the cap from the tube before it is positioned into the instrument holder.
2. For instruction for use regarding the automatic wash cycle, please refer to the Operator Manual of the instrument.

STORAGE

1. This product is ready to use and no further preparation is necessary. The unopened tube can be stored at 5 - 25°C until used or until the expiration date. After opening, store inside the instrument.
2. Improper storage will result in loss of efficacy. Do not use after expiration date.

WARNING AND PRECAUTIONS

1. Do not use if the tube is opened or damaged.
2. For in vitro use only.
3. Do not ingest the liquid.
4. Carefully follow the instructions for use. The manufacturer cannot be held responsible for any unauthorised or unqualified use of the product.
5. For professional use only. The product must be used only by adequately instructed personnel.
6. Unused tubes may be considered non-hazardous waste and be disposed of accordingly. For disposal of the used tubes follow the Laboratory Good Practices.
7. Once the **Loop Cleansing Solution** tube is placed inside the instrument, it should be daily checked for the liquid level and the content of carbon residues and it should be replaced if necessary (for further details refer to the Operator

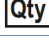
Manual of the instrument).

8. The tube should be changed at least on a weekly base.
9. **Loop Cleansing Solution** may show liquid opacity; this opacity is not to be considered a product defect and does not affect the product performance.
10. Do not use in case the package is open or damaged.

DISPOSAL

For disposal of the used **Loop Cleansing Solution** tubes follow the Laboratory Good Practices.

TABLE OF SYMBOLS

Symbol	Meaning
	Manufacturer
	Batch code
	Catalogue number
	Use by-date
	Temperature limits
	In Vitro Diagnostic Medical Device
	Consult the operating instructions supplied with the device or available in electronic format, and which can be identified by the e-IFU indicator on the packaging label
	Quantity (expressed in terms of numerical count)
Rx Only	For US Only Caution: US Federal Law restricts this device to sale by or on the order of a licensed practitioner

In case of serious incident, contact the manufacturer and the Competent Authority.

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