Copan

HPC127R02 Date 2021.11

**FLOQSwabs<sup>®</sup>**

**Instructions for Use**

**CE 0123**

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**Copan FLOQSwabs® in tube - Product Insert & How to Use Guide**
**INTENDED USE**

Copan FLOQSwabs® in tube are sterile and ready-for-use systems intended for clinical samples drawing. The swab is suitable for short contact with the human body. Product should be used as directed.

**SUMMARY AND PRINCIPLES**

One of the routine procedures in the diagnosis of infections involves the collection of swab samples. This can be accomplished using the Copan Swab.

Copan FLOQSwabs® are available with different size and shape of shaft and nylon tip for the different body collection sites. The FLOQSwabs® are provided into polypropylene labeled tube. A specific product line of Copan FLOQSwabs® is also available with the feature of free of amplifiable human DNA, DNase and RNase.

FLOQSwabs® are EO sterilized.

**TECHNICAL NOTE**

A slight yellowing of the tip is a well-known phenomenon. This could be due to many factors: to the type of raw material, to the product sterilization treatment, to the product natural aging. Therefore product yellowing is not necessarily indicative of product deterioration.

**PRECAUTIONS**

1. Carefully follow the user's instructions.
2. When using a device with plastic or wooden stick, the pressure applied during sampling shall be light since the stick material is breakable.
3. The fiber adhesion to the stick is tested for instantaneous sampling: a longer contact between the swab and the sampling area might cause the fiber to come out.
4. If the swab is submitted to a chemical or physical sterilizing or micro-biostatic process, its intended functioning could be compromised.
5. Product cannot be reused or re-sterilized.

**STORAGE**

Store the Copan Swab devices at 2-30°C. If the product is stored at a different temperature, it would not maintain the stated specifications.

**PRODUCT DETERIORATION**

The content of the unopened and undamaged units is guaranteed to be sterile.

Copan Swabs should not be used if (1) there is evidence of damage or contamination to the product, (2) the expiration date has passed, (3) the swab package is open or damaged, or (4) there are other signs of deterioration.





**DIRECTIONS FOR USE, SPECIMEN COLLECTION**

NOTE: Sterile gloves and protective clothing and eyewear should be worn when collecting and handling microbiology specimens

1. Open the tube and extract the swab
2. Collect the sample. During sampling, the swab end (tip) shall only come in contact with the suspected infection, so as to reduce contamination risks.

NOTE: Do not use excessive force, pressure or bending when collecting swab samples from patients as this may result in accidental breakage of the swab shaft. Swab shafts often exhibit diameter changes to facilitate different sampling requirements. Swab shafts may also have a molded breakpoint point designed for intentional breakage of the swab into a transport tube. In all circumstances when collecting swabs samples from patients, do not use excessive force, pressure or bending of the swab as this may result in accidental breakage of the swab shaft.

3. Process the swab according to internal laboratory procedure. Alternatively put the swab into its transport test tube.
4. Write the patient's name and data on the tube label.
5. Send the sample to the laboratory for immediate testing.

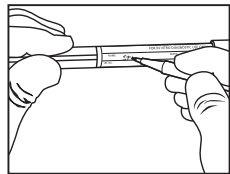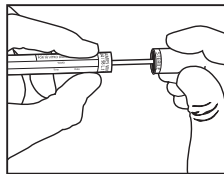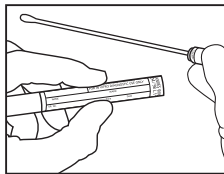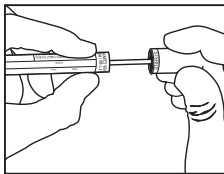Specimens collected for clinical investigations should be collected and handled following published manuals and guidelines.

Specific requirements for the shipment and handling of specimens should be in full compliance with state and federal regulations. Shipping of specimens within medical institutions should comply with internal guidelines of the institution.

**LIMITATIONS**

1. In the laboratory, wear latex gloves and other protection commensurate with universal precautions when handling clinical specimens. Observe Biosafety recommendations when handling or analyzing patient samples.
2. Condition, timing, and volume of specimen collected for clinical investigation are significant variables in obtaining reliable results. Follow recommended guidelines for specimen collection.
3. Copan Swab must be used by skilled personnel only.

**WARNINGS**

1. This product is for single use only; reuse may cause a risk of infection and/or inaccurate results.
2. Do not re-sterilize unused swabs.
3. Do not re-pack.
4. Not suitable for any other application than intended use.

5. The use of this product in association with a rapid diagnostic kit or with diagnostic instrumentation should be previously validated by the user.
6. Do not use if the swab is visibly damaged (i.e., if the swab tip or swab shaft is broken).
7. Do not use excessive force, pressure or bending when collecting swab samples from patients as this may result in accidental breakage of the swab shaft.
8. Directions for use must be followed carefully. The manufacturer cannot be held responsible for any unauthorized or unqualified use of the product.
9. To be handled by trained personnel only.
10. It must be assumed that all specimens contain infectious micro-organisms; therefore all specimens must be handled with appropriate precautions. After use, tubes and swabs must be disposed of according to laboratory regulations for infectious waste.

**WASTE DISPOSAL**

Unused swabs may be considered as non hazardous waste and disposed of accordingly.

Dispose of used swabs as well as any other contaminated disposable materials following procedures for infectious or potentially infectious products. It is the responsibility of each laboratory to handle waste and effluents produced according to their nature and degree of hazardousness and to treat and dispose of them (or have them treated and disposed of) in accordance with applicable regulations.

## Index of Symbols

Symbol / Simbolo / Simbolo / Symbol / Symbole / Símbolos / Symbol / Symbol / Symbol / Simbol / Symbol / Символ	Meaning / Significato / Signification / Bedeutung / Sens / Significado / Význam / Betyder / Betydning / Značenje / Betydelse / Значення
	Manufacturer / Fabricante / Fabricante / Hersteller / Fabricant / Fabricante / Výrobce / Producent / Produsent / Proizvođač / Tillverkare / Виробник
CE 0123	Identification number of notified body / Numero di identificazione dell'organismo notificato / Identificación del organismo notificado / Identifizierung der benannten Stelle / Identification de l'organisme notifié / Identificação do organismo notificado / Identifikační číslo notifikované osoby / Identifikationsnummer for det bemyndigede organ / Identifisering av godkjenningsorganet / Identifikacioni broj prijavljenog tela / Det anmälda organets identifieringskod / Ідентифікаційний номер уповноваженого органу
	Sterilized using ethylene oxide / Sterilizzato usando ossido di etilene / Esterilizado por óxido de etileno / Sterilisiert mit Äthylenoxid / Stérilisé à l'aide d'oxyde d'éthylène / Esterilizado utilizando óxido de etileno / Sterilizováno pomocí etylenoxidu / Steriliseret ved brug af ethylenoxid / Sterilisert ved bruk av etylenoksid / Sterilizovano korišćenjem etilen oksida / Steriliserad med etylenoxid / Стерилізований завдяки етиленоксиду
	Do not reuse / Non riutilizzare / No reutilizar / Nicht zur Wiederverwendung / Ne pas réutiliser / Não voltar a utilizar / Nepoužívejte znovu / Brug det ikke igen / Må ikke gjenbrukes / Nemojte ponovo koristiti / Återanvänd inte / Не використовувати повторно
	Catalogue number / Numero di catalogo / Número de catálogo / Bestellnummer / Référence du catalogue / Número do catálogo / Číslo katalogu / Katalognummer / Katalognr / Kataloški broj / Katalognummer / Номер за каталогом
	Temperature limitation / Limiti di temperatura / Límites de temperatura / Temperature Begrenzung / Limites de temperature / Limites de temperatura / Teplotní limity / Temperaturgrænser / Temperaturgrenser / Opseg temperature / Temperaturgränser / Температурні обмеження
	Use by / Utilizzare entro / Fecha de caducidad / Verwendbar bis / Utiliser jusque / Prazo de validade / Použijte do / Anvendes før / Må brukes innen / Upotrebiti do / Sista förbrukningsdag / Використати до
	Consult Instructions for Use / Consultare le istruzioni per l'uso / Consulte las instrucciones de uso / Gebrauchsanweisung beachten / Consulter les instructions d'utilisation / Consultar as instruções de utilização / Konzultujte návod k použití / Se brugsvejledningen / Se instruksjoner for bruk / Pročitajte uputstvo za upotrebu / Läs bruksanvisningen / Користуватись інструкціями по використанню
	Peel / Strappare per aprire / Desprender / Abziehen / Décoller / Destacar / Odrhňte a otevřete / Riv op for at åbne / Riv opp for å åpne / Ogulite / Dra för att öppna / Відкривати тут
	Batch code (Lot) / Codice del lotto (partita) / Código de lote (Lote) / Chargencode (Chagenbezeichnung) / Code de lot (Lot) / Código do lote (Lote) / Kód šarže (dávky) / Serienummer (parti) / Lot-nummer (parti) / Serijski (lot) broj / Partinummer (lotto) / Код серії (партії)
	Contains sufficient for <n> tests / Contenuto sufficiente per <n> test / Contenido suficiente para <n> pruebas / Ausreichend für <n> Tests / Contenu suffisant pour <n> tests / Contém o suficiente para <n> testes / Obsah dostatečný pro <n> testů / Indhold tilstrækkeligt til <n> prøver / Innhold tilstrekkelig for <n> prøver / Sadržaj dovoljan za   testova / Innehåller tillräckligt för <n> tester / Придатний для проведення <n> кількості випробувань
	Do not use if package is damaged / Non utilizzare in caso di confezionamento danneggiato / No utilizar en caso de paquete dañado / Bei Beschädigung der Verpackung nicht verwenden / Ne pas utiliser si l'emballage est abîmé / Não utilizar se a embalagem estiver danificada / Nepoužívejte v případě poškozeného obalu / Må ikke anvendes, hvis emballagen er beskadiget / Må ikke brukes hvis emballasjen er skadet / Nemojte koristiti ako je ambalaža oštećena / Använd inte vid skadat förpackning / Не використовувати, якщо упаковка пошкоджена

FLOQSwabs® are Copan Italia trademarks.

FLOQSwabs® are PATENTED WORLDWIDE No. PCT/EP2004/003392 and in the following countries: European Patent #1608268, US Patent #US8114027B2, Canadian Patent #2515205, Japanese Patent #2007-523663, Australian Patent #2004226798, New Zealand Patent #541560, Chinese Patent #101103931.

**Copan**

**Copan Italia S.p.A.**  
Via F. Perotti, 10  
25125 Brescia, Italy  
Tel +39 030 2687211  
Fax +39 030 2687250

Email: [info@copangroup.com](mailto:info@copangroup.com)  
Website: [www.copangroup.com](http://www.copangroup.com)

**North American Distributor:**

Copan Diagnostics Inc.  
26055 Jefferson Avenue  
Murrieta, CA 92562, USA  
Tel: 951-696-6957  
Fax: 951-600-1832

E-mail: [customerservice@copanusa.net](mailto:customerservice@copanusa.net)  
Website: [www.copanusa.com](http://www.copanusa.com)