


Analytical Performance for OC-FCa Reagent



Repeatability and precision

Samples	Mean (µg/g)	Repeatability		Between-Run		Between-Day		Precision	
		SD	CV	SD	CV	SD	CV	SD	CV
Control LV1	252.7	2.8	1.1%	2.2	0.9%	2.9	1.2%	4.6	1.8%
Control LV2	496.0	4.1	0.8%	2.3	0.5%	4.4	0.9%	6.5	1.3%
Faecal Sample 1	53.0	3.0	5.6%	1.5	2.8%	0.7	1.4%	3.4	6.4%
Faecal Sample 2	284.3	2.6	0.9%	1.3	0.4%	2.7	0.9%	3.9	1.4%


Dilution Linearity

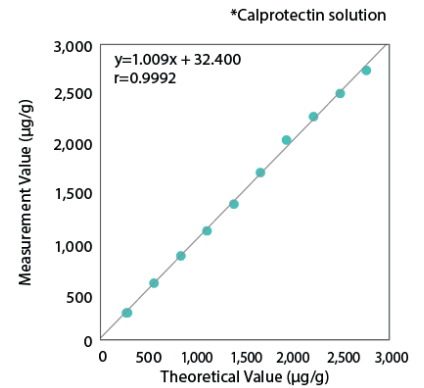

Comparison (FEIA) vs. OC-Sensor PLEDIA

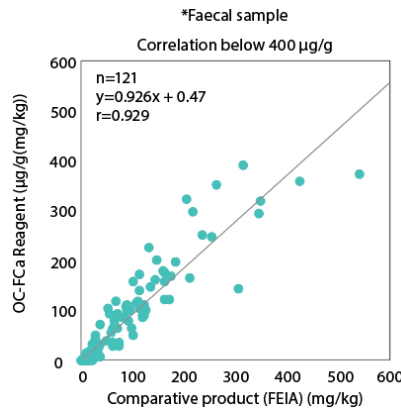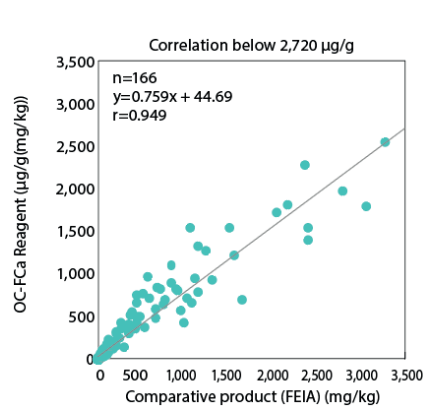

Comparison OC-Sensor PLEDIA vs. Ceres

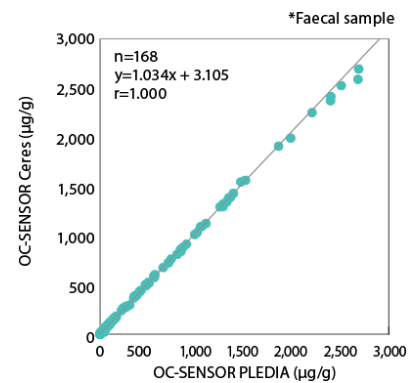

Cut-off value	Positive agreement	Negative agreement	Overall agreement
50 µg/g	95.6% (108/113)	96.9% (62/64)	96.0% (170/177)

MD V.1.0 / 2024-07-23

Distributed by:



Mast Diagnostica GmbH
 Feldstraße 20
 DE-23858 Reinfeld

Tel: +49 (0) 4533 2007 0
 Fax: +49 (0) 4533 200768
 E-Mail: mast@mast-diagnostica.de

Manufacturer:



EIKEN CHEMICAL CO., LTD.
 4-19-9, Taito-ku, Tokyo 110-8408, Japan

Advena Ltd.
 Tower Business Centre, 2nd Flr., Tower Street
 Swatar, BKR 4013 Malta

