# Vitassay

## Real-Time PCR Kits

### Vitassay qPCR Dengue 1+2+3+4

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- LightCycler 480II (Roche)
- Cobas Z480 (Roche)
- 7500 Fast Real-Time PCR System (Applied Biosystems) <sup>II</sup>
- StepOne™ Real-Time PCR System (Applied Biosystems) <sup>II</sup>
- CFX96™ Real-Time PCR Detection System (Bio-Rad)
- AriaMx Real-Time PCR System (Agilent Technologies)
- DTLite Real-Time PCR System (DNA-Technology)
- DTPrime Real Time Detection Thermal Cycler (DNA-Technology)
- Rotor-Gene® Q (Qiagen)<sup>I</sup>
- SmartCycler® (Cepheid)<sup>I</sup>

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Termocicladores con bloque de bajo perfil	Termocicladores con bloque de alto perfil
<b>Agilent Technologies</b>	<b>Abbott</b>
AriaMx Real-Time PCR System	Abbott m2000 Real-Time System
<b>Applied Biosystems</b>	<b>Applied Biosystems</b>
7500 Fast Real-Time PCR System	7300 Real-Time PCR System
7500 Fast Dx Real-Time PCR System	7500 Real-Time PCR System
QuantStudio™ 12K Flex 96-well Fast	7900 HT Real-Time PCR System
QuantStudio™ 6 Flex 96-well Fast	ABI PRISM 7000
QuantStudio™ 7 Flex 96-well Fast	ABI PRISM 7700
QuantStudio™ 5 Real-Time PCR System	QuantStudio™ 12K Flex 96-well
QuantStudio™ 3 Real-Time PCR System	QuantStudio™ 6 Flex 96-well
StepOne Plus™ Real-Time PCR System	QuantStudio™ 7 Flex 96-well
StepOne™ Real-Time PCR System	QuantStudio™ 5 Real-Time PCR System
ViiA™ 7 Fast Real-Time PCR System	QuantStudio™ 3 Real-Time PCR System
<b>BIONEER</b>	ViiA™ 7 Real-Time PCR System
Exicycler™ 96	<b>Analytik Jena Biometra</b>
<b>Bio-Rad</b>	TOptical
CFX96™ Real-Time PCR Detection System	qTOWER 2.0
Mini Opticon™ Real-Time PCR Detection System	<b>BIONEER</b>
<b>Cepheid</b>	Exicycler™ 96
SmartCycler®	<b>Bio-Rad</b>
<b>Qiagen</b>	CFX96™ Deep Well Real-Time PCR
Rotor-Gene® Q	iCycler iQ™ Real-Time PCR
<b>Roche</b>	iCycler iQ™ 5 Real-Time PCR
LightCycler @480 Real-Time PCR System	MyiQ™ Real-Time PCR Detection System
LightCycler @96 Real-Time PCR System	MyiQ™ 2 Real-Time PCR Detection System
Cobas z480 Analyzer	<b>Cepheid</b>
	SmartCycler®
	<b>DNA-Technology</b>
	DTlite Real-Time PCR System *
	DTprime Real-time Detection Thermal Cyclor *
	<b>Eppendorf</b>
	Mastercycler™ ep <i>realplex</i>
	<b>Qiagen</b>
	Rotor-Gene® Q
	<b>Stratagene/ Agilent Technologies</b>
	Mx3000P™ Real Time PCR System
	Mx3005P™ Real Time PCR System

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## Intended use

Vitassay qPCR Dengue 1+2+3+4 allows the detection and differentiation of Dengue virus serotypes (1, 2, 3, y/o 4) by real-time PCR in clinical samples. The product is intended for use in the diagnosis of Dengue virus infections alongside clinical data of the patient and other laboratory tests outcomes.

## References

Vitassay qPCR Dengue 1+2+3+4 8x8-well strip, low profile	7041006
Vitassay qPCR Dengue 1+2+3+4 8x8-well strip, high profile	7042006

## Reagents provided

In references 7041006 and 7042006:

Reference	Reagent/Material	Color	Amount
7041S006A/ 7042S006A	Dengue 1+4 strips low/high profile	-	4 x 8-well strip
7041S006B/ 7042S006B	Dengue 2+3 strips low/high profile	-	4 x 8-well strip
7C006	Dengue 1+2+3+4 Positive Control	red	1 vial
7001A	PCR grade water	white	1 vial x 1 mL
7002B	Resuspension buffer	green	1 vial x 1,8 mL
7003N	Negative control	yellow	1 vial x 1 mL
7004O	Optical caps	-	8 x 8-cap strip

## Material and equipment required, not provided

- RNA extraction kit
- Real-time PCR instrument (thermocycler) (Attached I)
- Centrifuge for 1.5 ml tubes
- Vortex
- Micropipettes (1-20 µl, 20-200 µl)
- Filter tips
- Powder-free disposal gloves

## Transport and storage

- The reagents and the test can be shipped and stored at 2-40°C until expiration date stated in the label.
- The resuspended positive control should be stored at -20°C. To avoid repeated freeze/thaw cycles, it is recommended to distribute the content in different aliquots.
- Keep all reagents in the darkness.

## Summary

Dengue is a mosquito-borne viral disease that has rapidly spread in recent years. Dengue virus is transmitted by female mosquitoes mainly of the species *Aedes aegypti* and, to a lesser extent, *Ae. Albopictus*. One study of the prevalence of dengue, estimates that 3.9 billion people, in 128 countries, are at risk of infection with dengue viruses.

Dengue fever can be caused by any of four genetically related but antigenically distinct dengue virus (DENV) serotypes (DENV-1, DENV-2, DENV-3, and DENV-4).

Many people may experience no symptoms during a mild case of dengue fever. When symptoms do occur, they usually begin four to 10 days after you are bitten by an infected mosquito. Signs and symptoms of dengue fever most commonly include high fever, headaches, muscle, bone and joint pain, pain behind your eyes. You might also experience widespread rash, nausea and vomiting

and rarely, minor bleeding from your gums or nose. In some cases, symptoms worsen and can become life-threatening. Blood vessels often become damaged and leaky. And the number of clot-forming cells (platelets) in your bloodstream drops.

Multiple factors have been suggested to contribute to severe dengue such as secondary infections, age, viral load as well infecting serotype and genotype. Findings illustrate the uniqueness of each serotype in producing epidemics and severe disease and underscore the importance of long-term surveillance of dengue serotyping in understanding the epidemiology of these viruses. It is for that a Dengue virus serotyping is essential.

Dengue can be diagnosed by isolation of the virus, by serological tests (antigen and antibodies detection), or by molecular methods. Methods such as one-step real time RT-PCR are now widely used to detect dengue viral genes in acute-phase serum samples. This detection coincides with the viremia and the febrile phase of illness onset. It's important to consider that serological methods have cross reactivity with other flaviviruses including West Nile virus (WNV), St. Louis encephalitis virus (SLE), Japanese encephalitis virus (JEV) and yellow fever virus (YFV).

### Test principle

Vitassay qPCR Dengue 1+2+3+4 is based on the real-time amplification of specific conserved fragments of the *NS5* gene (Dengue serotype 1 (DEN-1)), *envelope* gene (Dengue virus serotype 2 (DEN-2)), *prM* gene (Dengue serotype 3 (DEN-3)), and *NS2A* gene (Dengue serotype 4 (DEN-4)). The viral RNA extracted is transcribed into cDNA using a specific primer by reverse transcription step followed immediately in the same well by polymerase chain reaction. The presence of the virus is detected by an increase in observed fluorescence during the reaction upon hydrolysis of the fluorescent probe.

Vitassay qPCR Dengue 1+2+3+4 is a ready-to-use test which contains in each well all the necessary reagents for real-time PCR assay in a stabilized format. In addition, an internal control allows the detection of a possible reaction inhibition. Each kit includes two kind of strips and each one corresponds to one different assay. The first strip contains the multiplex reaction mix for the detection of DEN-1 and/or DEN-4. The second strip contains the multiplex reaction mix for the detection of DEN-2 and/or DEN-3. The detection channels of the target sequences are described below:

Target	Detection Channel
DEN-1 y/o DEN-3	FAM
DEN-2 y/o DEN-4	ROX
Internal Control (IC)	HEX, VIC, or JOE (depending on the equipment used)

### Precautions

- For professional *in vitro* diagnostic use
- Do not use after expiration date.
- Do not mix components from different kits and/or lots.
- Do not use if package is open or damaged.
- Do not use the kit if the desiccant from the different reagents is not present or broken or if the foil has been broken or damaged.
- Protect the kit against humidity. Prolonged exposure to humidity may affect product performance.
- Follow Good Laboratory Practices. Wear protective clothing, use disposal gloves, goggles, and mask.
- Do not eat, drink, or smoke in the working area.
- The laboratory process must be one-directional, it should begin in the Extraction Area and then move to the Amplification and Detection Areas. Do not return samples, equipment, and reagents to the area in which the previous step was performed.
- Specimens and reagents/materials that have been exposed to them must be treated as potentially infectious. Take necessary precautions during the collection, storage, treatment, and disposal of samples.
- Regular decontamination of commonly used equipment is recommended, especially micropipettes and work surfaces.

### Procedures

#### Specimen collection, transport, and RNA extraction

For pre-treatment and nucleic acid isolation, it is recommended to use your optimized manual or automatic system compatible with real-time PCR technology. For the sample's pretreatment follow the instructions for use of the extraction kit used. The assay has been validated with the following extraction kits:

- Maxwell® 16 Viral Total Nucleic Acid Purification Kit (Promega)
- QIAamp Viral RNA Mini Kit, utilizando QIAcube instrument (Qiagen)
- Invisorb® Spin Universal Kit (Invitex)

### Positive control preparation

Reconstitute the lyophilized Dengue 1+2+3+4 Positive Control (red tube) with the 200 µL of PCR grade water (white tube). To ensure a complete resuspension, vortex the tube thoroughly. After first use, dispense into aliquots to avoid multiple freeze-thaw cycles, and store them at -20°C.

This component contains high copies number template and is a very significant contamination risk. Therefore, we recommend open and manipulate it in a separate laboratory area away from the other components.

### Reaction setup

- Prepare the number of reactions required including samples and controls. Remember that one positive and one negative control must be included in each run.
- Peel off protective aluminium seal from the strips.
- Pipette 15 µL of Resuspension buffer (green tube) and add them into each well.
- Pipette 5 µL of RNA sample, negative and positive controls into each well.
- Cover the wells with the caps provided. Spin down briefly (optional).
- Place the strips in the Real-time PCR instrument.

### Programme your thermocycler

Set your thermocycler following the conditions below:

Step	Temperature	Time	Cycles
Reverse transcription	45°C	15 min	1
Initial denaturation	95°C	2 min	1
Denaturation	95°C	10 sec	45
Annealing/Extension (Data collection*)	60°C	50 sec	

Set the fluorescence data collection during the extension step (\*) through the channels FAM (Dengue 1 and 3), ROX (Dengue 2 and 4), and HEX, JOE, or VIC channels (Internal Control (IC)). If you use the Applied Biosystems 7500 Fast, Applied Biosystems StepOne™, or the Stratagene Mx3005P™ check that passive reference option ROX is none. (Attached II).

### Analysis and interpretation of results

The analysis of the results is done by the software itself of the used real-time PCR system following manufacturer's instructions.

For a valid diagnostic test run, the following control conditions must be met:

#### Positive control

The positive control used in each run must show an amplification curve for FAM (Dengue 1 and 3) and ROX (Dengue 2 and 4) channels, which validates the reaction.

#### Negative control

The negative controls included in each run must show the absence of signal in FAM and ROX channels, which validates the reaction.

The experiment seems to be failed if there is signal of amplification in negative control or absence of signal in the positive well. The assay should be repeated.

The result interpretation is summarized in the following table:

Dengue 1 (FAM)	Dengue 4 (ROX)	Control Interno	Control Negativo	Control Positivo	Interpretación
+	+	+/-	-	+	Dengue serotypes 1 and 4 Positive
-	-	+	-	+	Dengue serotypes 1 and 4 Negative
+	-	+/-	-	+	Dengue serotype 1 Positive and Dengue serotype 4 Negative
-	+	+/-	-	+	Dengue serotype 4 Positive and Dengue serotype 1 Negative
+	+	+	+	+	Experiment fail
-	-	-	-	-	Experiment fail

Dengue 2 (ROX)	Dengue 3 (FAM)	Control Interno	Control Negativo	Control Positivo	Interpretación
+	+	+/-	-	+	Dengue serotypes 2 and 3 Positive
-	-	+	-	+	Dengue serotypes 2 and 3 Negative
+	-	+/-	-	+	Dengue serotype 2 Positive and Dengue serotype 3 Negative
-	+	+/-	-	+	Dengue serotype 3 Positive and Dengue serotype 2 Negative
+	+	+	+	+	Experiment fail
-	-	-	-	-	Experiment fail

**(+) Positive:** Amplification signal

**(-) Negative:** No amplification signal

If the negative samples do not show a positive result for the internal control, they should be retested from the diluted original sample 1:10 or the nucleic acid extraction must be repeated due to possible problems caused by PCR inhibitors.

## Quality Control

To confirm the appropriate performance of the molecular diagnostic technique, an Internal Control (IC) is included in each reaction. Besides, a positive and a negative control must be included in each assay to interpret the results correctly.

## Performance evaluation

### Clinical sensitivity and specificity

Vitassay qPCR Dengue 1 + 2 + 3 + 4 was evaluated with the programmes: QCMD 2017 Dengue Virus EQA programme (DENVRNA17) and QCMD 2018 Dengue Virus EQA programme (DENVRNA18). Both programmes consisted of panels of 10 samples dissolved in transport medium positive or not to the studied pathogen (in total 20 samples). In addition, samples from the DENVRNA17 panel were studied with the RealStar® Dengue RT-PCR Kit (Altona Diagnostics) and the results were compared with those obtained with Vitassay qPCR Dengue 1 + 2 + 3 + 4. The same results were obtained with both kits.

Vitassay qPCR Dengue 1 + 2 + 3 + 4 correctly detected all samples from both panels. These results indicate that Vitassay qPCR Dengue shows a high sensitivity and specificity to detect the Dengue virus.

### Analytical sensitivity

The analytical sensitivity was determined by analysis of 10-fold dilution series of Dengue 1+2+3+4 virus templates ranging from  $10^7$  to  $10^1$  copies/reaction. This assay has a detection limit (LoD) of  $\geq 10$  RNA copies per reaction.

## Analytical specificity

The analytical specificity of Dengue 1+2+3+4 virus was tested within the panel of following virus, where no cross-reactivity was observed between any of the species.

Cross-reactivity assay		
Chikungunya virus S27 Petersfield	West Nile virus strain H160/99	Yellow Fever virus strain 17D
Zika virus strain MR 766	West Nile virus strain Heja	
St Louis Encephalitis virus strain 17D	West Nile virus strain Ug37	

## Analytical reactivity

The reactivity of Vitassay qPCR Dengue 1+2+3+4 for Dengue virus serotypes was confirmed by the real time amplification using

- Dengue 1 virus strain Hawaii
- Dengue 2 virus strain New Guinea C
- Dengue 3 virus strain H87
- Dengue 4 virus strain H241

## Compatibles real-time PCR equipment

Vitassay qPCR Dengue 1+2+3+4 has been validated on the following equipment:

- LightCycler 480II (Roche)
- Cobas Z480 (Roche)
- 7500 Fast Real-Time PCR System (Applied Biosystems) <sup>II</sup>
- StepOne™ Real-Time PCR System (Applied Biosystems) <sup>II</sup>
- CFX96™ Real-Time PCR Detection System (Bio-Rad)
- AriaMx Real-Time PCR System (Agilent Technologies)
- DTLite Real-Time PCR System (DNA-Technology)
- DTPPrime Real Time Detection Thermal Cycler (DNA-Technology)
- Rotor-Gene® Q (Qiagen)<sup>I</sup>
- SmartCycler® (Cepheid)<sup>I</sup>

I: For Rotor-Gene® Q and SmartCycler® thermocyclers the product should be reconstituted following the procedure and transferred into specific Rotor-Gene® and/or SmartCycler tubes.

II: When using the Applied Biosystems 7500 Fast with strips, it is recommended to place a plate holder for the tubes (Ref. PN 4388506).

## Limitations

- This test provides a presumptive diagnosis of Dengue 1, 2, 3, and 4 virus infection. All obtained results must be interpreted together with other clinical information and laboratory findings available to the physician.
- This assay was tried with serum samples. The use of other samples has not been established.
- The quality of the test depends on the quality of the sample; proper RNA from clinical samples must be extracted. Unsuitable collection, storage and/or transport of specimens may give false negative results.
- Extremely low levels of target below the limit of detection may be detected, but results may not be reproducible.
- There is a possibility of false positive results due to cross-contamination by Dengue 1, 2, 3, and 4 viruses either samples containing high concentrations of target RNA or contamination from PCR products from previous reactions.

## Attached I. Compatibility of the most common real-time PCR equipment

The most common thermocyclers are listed in the following table separated by tube type. If you do not find your thermocycler, please contact with your supplier.

Low profile Block Thermocyclers	High profile Block Thermocyclers
<b>Agilent Technologies</b>	<b>Abbott</b>
AriaMx Real-Time PCR System	Abbott m2000 Real-Time System
<b>Applied Biosystems</b>	<b>Applied Biosystems</b>
7500 Fast Real-Time PCR System	7300 Real-Time PCR System
7500 Fast Dx Real-Time PCR System	7500 Real-Time PCR System
QuantStudio™ 12K Flex 96-well Fast	7900 HT Real-Time PCR System
QuantStudio™ 6 Flex 96-well Fast	ABI PRISM 7000
QuantStudio™ 7 Flex 96-well Fast	ABI PRISM 7700
QuantStudio™ 5 Real-Time PCR System	QuantStudio™ 12K Flex 96-well
QuantStudio™ 3 Real-Time PCR System	QuantStudio™ 6 Flex 96-well
StepOne Plus™ Real-Time PCR System	QuantStudio™ 7 Flex 96-well
StepOne™ Real-Time PCR System	QuantStudio™ 5 Real-Time PCR System
ViiA™ 7 Fast Real-Time PCR System	QuantStudio™ 3 Real-Time PCR System
<b>BIONEER</b>	ViiA™ 7 Real-Time PCR System
Exicycler™ 96	<b>Analytik Jena Biometra</b>
<b>Bio-Rad</b>	TOptical
CFX96™ Real-Time PCR Detection System	qTOWER 2.0
Mini Opticon™ Real-Time PCR Detection System	<b>BIONEER</b>
<b>Cepheid</b>	Exicycler™ 96
SmartCycler®	<b>Bio-Rad</b>
<b>Qiagen</b>	CFX96™ Deep Well Real-Time PCR
Rotor-Gene® Q	iCycler iQ™ Real-Time PCR
<b>Roche</b>	iCycler iQ™ 5 Real-Time PCR
LightCycler® 480 Real-Time PCR System	MyiQ™ Real-Time PCR Detection System
LightCycler® 96 Real-Time PCR System	MyiQ™ 2 Real-Time PCR Detection System
Cobas z480 Analyzer	<b>Cepheid</b>
	SmartCycler®
	<b>DNA-Technology</b>
	DTlite Real-Time PCR System
	DTprime Real-time Detection Thermal Cyclers
	<b>Eppendorf</b>
	Mastercycler™ ep <i>realplex</i>
	<b>Qiagen</b>
	Rotor-Gene® Q
	<b>Stratagene/ Agilent Technologies</b>
	Mx3000P™ Real Time PCR System
	Mx3005P™ Real Time PCR System

\* See Attached III to configure exposure settings.

## Attached II. Detection channels of most common real time PCR equipment

The fluorescence detection channels of some of most common real-time PCR thermocyclers are specified in the following table:

Thermocycler	Vitassay Channel	Detection Channel	Observations
<b>Bio-Rad CFX96™</b>	FAM	FAM	
	HEX	HEX	
	ROX	ROX	
	Cy5	Cy5	
<b>Applied Biosystems ABI 7500</b>	FAM	FAM	Passive reference option ROX is none.
	HEX	VIC	
	ROX	ROX	
	Cy5	Cy5	
<b>Roche Lightcycler®480II</b>	FAM	465/510	Color Compensation required
	HEX	533/580	
	ROX	533/610	
	Cy5	618/660	
<b>Cepheid Smartcycler®</b>	FAM	Channel 1	
	HEX	Channel 2	
	ROX	Channel 3	
	Cy5	Channel 4	
<b>Abbott m2000rt</b>	FAM	FAM	
	HEX	VIC	
	ROX	ROX	
	Cy5	Cy5	
<b>Agilent Technologies Mx3000P™ Mx3005P™</b>	FAM	FAM	Passive reference option ROX is none.
	HEX	HEX	
	ROX	ROX	
	Cy5	Cy5	
<b>Agilent Technologies AriaMx</b>	FAM	FAM	
	HEX	HEX	
	ROX	ROX	
	Cy5	Cy5	
<b>Qiagen Rotor-Gene® Q</b>	FAM	Green	In the Channel Setup, click on the "Gain Optimisation" button and then go to "Optimise Acquiring". The fluorescence Target Sample Range must be between 5 and 10 FI for each channel. Also select the option "Perform Optimisation Before 1st Acquisition".
	HEX	Yellow	
	ROX	Orange	
	Cy5	Red	
<b>BIONEER Exicycler™ 96</b>	FAM	FAM	
	HEX	JOE	
	ROX	ROX	
	Cy5	Cy5	

**Attached III. Optical measurement exposure setting**

The exposure values of some thermocyclers must be adjusted for proper operation. Set exposure values as follows:

<b>Thermocycler</b>	<b>Vitassay channel</b>	<b>Exposure values</b>
<b>DTlite Real-Time PCR System (DNA-Technology)</b>	FAM	500
	HEX	500
	ROX	500
	Cy5	500
<b>DTprime Real-time Detection Thermal Cycler (DNA-Technology)</b>	FAM	500
	HEX	1000
	ROX	1000
	Cy5	1000

---

**Bibliography/Bibliografía**

1. G.A. Santiago et al. Analytical and clinical performance of the CDC real time RT-PCR assay for detection and typing of dengue virus. *PLOS Neglected Tropical Diseases* 2013; 7(7):e2311.
2. Drosten C. et al. Rapid detection and quantification of RNA of Ebola and Marburg viruses, Lassa virus, Crimean-Congo hemorrhagic fever virus, Rift Valley fever virus, dengue virus, and yellow fever virus by real-time reverse transcription-PCR. *Journal of Clinical Microbiology* 2002; 40(7): 2323-2330.
3. Dumoulin A. et al. Pan-dengue virus detection by PCR for travelers returning from the tropics. *Journal of Clinical Microbiology* 2008; 46(9): 3104-3106.
4. Liu J. et al. Development of a TaqMan Array Card for Acute-Febrile-Illness Outbreak Investigation and Surveillance of Emerging Pathogens, Including Ebola Virus. *Journal of Clinical Microbiology* 2016; 54(1): 49-58.
5. Mardekian S.K. and Roberts A.L. Diagnostic Options and Challenges for Dengue and Chikungunya Viruses. *BioMed Research International* 2015: 834371.
6. Centers for Disease Control and Prevention. Dengue (<http://www.cdc.gov/dengue/>).
7. World Health Organization. Dengue (<http://www.who.int/topics/dengue/en/>).
8. Fields BN, Knipe DM, Howley PM. Alphaviruses. In: *Fields Virology*. 3 rd ed. Philadelphia: Lippincott-Raven Publishers; 1996;1: 858-98.
9. Y. Chee-Fu et al. Dengue Serotype-Specific Differences in Clinical Manifestation, Laboratory Parameters and Risk of Severe Disease in Adults, Singapore. *Am J Trop Med Hyg.* 2015; 92(5): 999–1005.
10. N. Ananda et al. Serotype-specific dengue virus circulation and dengue disease in Bangkok. *Am. J. Trop. Med. Hyg.*, 68(2), 2003, 191–202.

## Trademarks

CFX™ and IQ5™ are registered trademarks of Bio-Rad Laboratories.

ABI®, QuantStudio™ and ViiA™ are registered trademarks of Thermo Fisher Scientific Inc.

LightCycler® is a registered trademark of Roche.

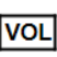
Mx3000P™ and Mx3005P™ are registered trademarks of Agilent Technologies.

Mastercycler™ is a registered trademark of Eppendorf.

Rotor-Gene® Q is a registered trademark of Qiagen.

SmartCycler® is a registered trademark of Cepheid.

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